McKESSON

Wrist Splint with Abducted Thumb

For use with MFR #'s 155-81-87302, 155-81-87303, 155-81-87305, 155-81-87307
155-81-87308, 155-81-87312, 155-81-87313, 155-81-87315, 155-81-87317, 155-81-87318.

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INDICATIONS

Treatment of deQuervains syndrome; Scaphoid injuries; Gamekeeper's thumb; and wrist and thumb sprains and strains.

PRECAUTIONS

- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

DIRECTIONS FOR USE

1. Loosen straps and slip hand into support with thumb resting in abduction stay.
2. The palmar stay should be resting on palmar aspect of the hand. Stay is malleable and may be contoured to fit individual patient needs.
3. Wrap and secure thumb strap.
4. Starting at the hand, feed straps through "D" rings and secure, working up the arm.

CLEANING INSTRUCTIONS

Hand wash liner in warm water using mild soap, rinse thoroughly. AIR DRY. NOTE: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

FOR SINGLE PATIENT USE ONLY.

Not made with natural rubber latex.

Questions? Call 1-800-777-4908

Technical Support? Call 1-800-336-6569

Distributed by McKesson Medical-Surgical Inc.

Richmond, VA 23233

PVN A0520

PN 13-5510 Rev A

Made in Taiwan

Symbol Definitions	
Symbol	Definition
	Consult instructions for use
	Caution, consult accompanying documents

NOTICE: While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through the use of this product.

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Distribuido por McKesson Medical-Surgical Inc.