Thumb Spica Splints

For use with MFR #'s 155-79-87113, 155-79-87114, 155-79-87117, 155-79-87118.

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INDICATIONS

Soft tissue injury to the thumb, Arthritis, Tendinitis, deQuervains Syndrome.

PRECAUTIONS

- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

DIRECTIONS FOR USE

- NOTE: Splint may be trimmed with scissors. Apply as noted below.
1. Detach forearm wrist straps.
 2. Slide thumb and forearm into splint, and secure thumb strap.
 3. Wrap and attach forearm strap then wrist strap. Straps should be snug but not restrict circulation.

CLEANING INSTRUCTIONS




Hand or machine wash in cold water with mild soap. Air dry. Do not bleach.

FOR SINGLE PATIENT USE ONLY.

Not made with natural rubber latex.

Questions? Call 1-800-777-4908
 Technical Support? Call 1-800-336-6569

Distributed by McKesson Medical-Surgical Inc.
 Richmond, VA 23233
 PVN A0520
 PN 13-5340 Rev A
 Made in Mexico

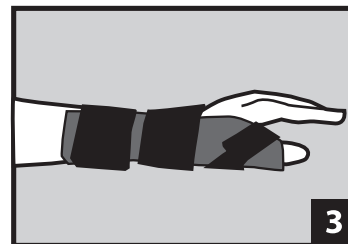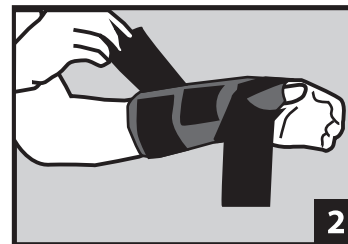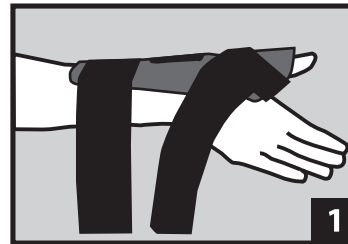Symbol Definitions	
Symbol	Definition
	Consult instructions for use
	Caution, consult accompanying documents

NOTICE: While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability, and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through the use of this product.

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Distribuido por McKesson Medical-Surgical Inc.