

MCKESSON

Toilet Safety Frame


Provides a total weight bearing capacity of 300 lbs (136 kg).

For use with MFR#'s 146-RTL12000.

INSTALLATION INSTRUCTIONS

WEIGHT CAPACITY: 300 LBS (136 KG)

1. Detach seat by first removing toilet seat bolts.
2. Place the rail mounting flange on the toilet and align the holes. Ensure tube on bracket is closest to the tank (Fig. 1).
3. Place toilet seat and align holes (Fig. 1).
4. Insert bolts through toilet seat and bracket using original nuts and secure.
5. Assemble safety frame by inserting short tube into arm tube as shown in Fig. 2.
6. Slide assist rails into bracket and secure them with built-in snap buttons. (Fig. 2)
7. Set rail to desired width and height using the built-in snap buttons. Make sure all built-in snap buttons are secured before use of product.
8. Use both hands whenever lowering or raising yourself from the toilet seating surface.

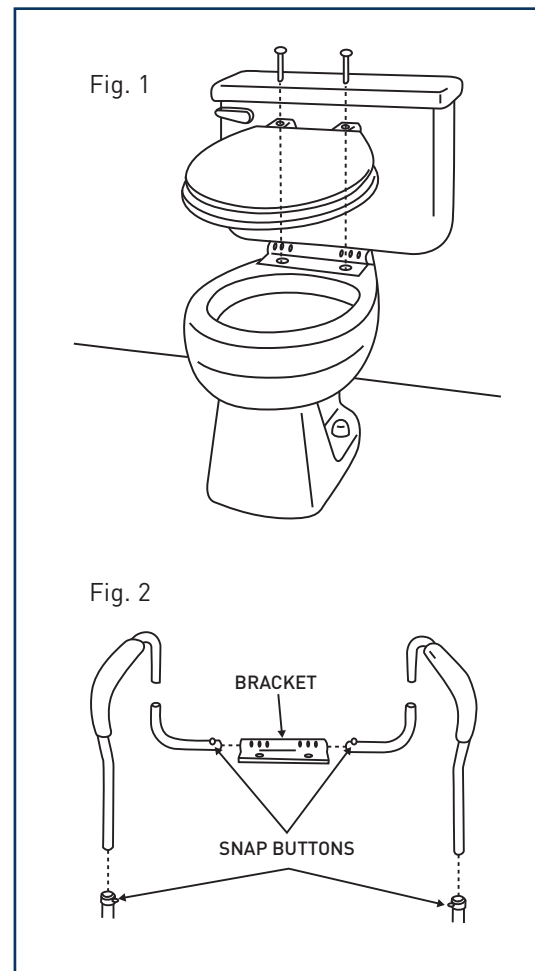

Technical support, replacement parts and warranty information? Call 1-866-694-5085. General product questions? Call 1-800-777-4908

Satisfaction Guaranteed

For complete details, please visit
mms.mckesson.com/mckesson-brands.

Distributed by McKesson Medical-Surgical Inc.
Richmond, VA 23233
PVN A0618
Made in China