

Multi-Enzymatic Cleanser

Directions for Use

1. **Manual Cleaning:** Use ½ to 1 ounce of McKesson Multi-Enzymatic Cleanser per one gallon water for general purpose cleaning. Additional amounts may be required for hard-to-remove matter. The enzymes selected for McKesson Multi-Enzymatic Cleanser have an optimum efficiency when used at a temperature of 95°F (35°C). It is recommended that these products be used between 68-122°F (20-50°C). The enzymes in McKesson Multi-Enzymatic Cleanser may become ineffective in water temperatures exceeding this level. McKesson Multi-Enzymatic Cleanser will begin breaking down organic matter immediately. A minimum soak time of 2 minutes is recommended. Hard-to-remove matter may possibly require additional soak time.
2. **Automatic Cleaning Units:** Follow recommendations under **Manual Cleaning** except that endoscope reprocessors may require less McKesson Multi-Enzymatic Cleanser. For endoscope reprocessors, ultrasonic units and washer sterilizers/decontaminators, process instruments according to manufacturer's instructions.
3. Thoroughly rinse McKesson Multi-Enzymatic Cleanser from instruments making sure to aspirate rinse water through all channels, box locks, lumens, etc.
4. Once instruments have dried, sterilization or disinfection may follow according to appropriate procedures.
5. Fresh McKesson Multi-Enzymatic Cleanser solution should be prepared daily or more frequently if solution is visibly soiled. Used solution may be disposed of into sewage disposal system flushing with large quantities of water and in accordance with federal, state or local regulations.
6. **DO NOT** add other chemicals such as bleaches, peroxides or other detergents which could destroy the effectiveness of McKesson Multi-Enzymatic Cleanser.

Satisfaction Guaranteed

For complete details, please visit
mms.mckesson.com/mckesson-brands.

Never Compromise on CareSM

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them—making it even easier for you to provide the best care for your patients.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.