

MCKESSON

Raised Toilet Seat with dual locks | plastic frame


Provides a total weight bearing capacity of 400 lbs (181 kg).

For use with MFR# 146-RTL12064.


INSTALLATION INSTRUCTIONS

Weight Capacity: 400 lbs (181 Kg)

1. Remove the product from carton and read directions carefully before installing raised toilet seat.
2. Remove the toilet seat and lid from the toilet.
3. Place the collar of the Handscrew into the semi-circle cut-out on the Locking Clamp (Figure 1).

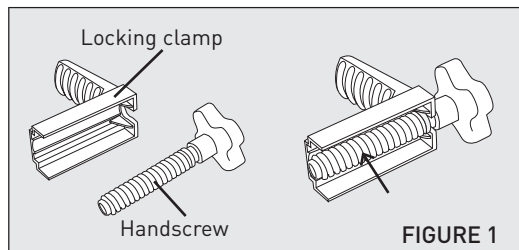

4. Slide the locking clamp into the void on the bottom of the toilet seat. Turn the handscrew clockwise for two rotations to attach the locking clamp to the toilet seat (Figure 2).

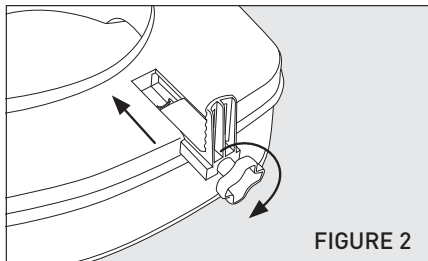

5. Place raised toilet seat onto the bowl and position in a secure fashion. Be sure the two locking brackets are on both sides of the bowl wall and that the seat is flush to the bowl.
6. Turn the locking brackets clockwise until tight. Hand tighten only. Do not use tools as threads on adjustment bracket may strip.

7. Check for secure fit routinely before using product. User's weight should be centered over toilet seat.

REMOVAL INSTRUCTIONS

1. Rotate knobs counter-clockwise until seat is loose.
2. Lift front of raised toilet and pull forward to move.

CLEANING INSTRUCTIONS

1. Use a non-abrasive detergent or household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. **DO NOT USE AN ABRASIVE CLEANER OR CLOTH** with this product, as damage may occur.

CAUTION

1. Always be sure that the seat is correctly and securely locked in place before using.
2. Be sure that adjustment knobs on sides of seat are tight.
3. This product is designed for use with most common types of toilets. It will not fit all toilet types; be sure that the unit is correctly installed, fits properly and is stable before use.

Technical support, replacement parts and warranty information? Call 1-866-694-5085. General product questions? Call 1-800-777-4908

Satisfaction Guaranteed

For complete details, please visit
mms.mckesson.com/mckesson-brands.

Distributed by McKesson Medical-Surgical Inc.
Richmond, VA 23233
PVN A0618
Made in Italy