

# McKESSON

## Round Handle Cane chrome

**LIGHTWEIGHT | ADJUSTABLE | VINYL CONTOURED GRIP**  
Provides a total weight bearing capacity of 300 lbs (136 kg).  
For use with MFR# 146-10302-6.

### SAFETY SUMMARY

To ensure the safe operation of McKesson Canes, these instructions **MUST** be followed.

### GENERAL WARNINGS

DO NOT use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment-otherwise, injury may occur.


Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately. McKesson Canes can provide ambulatory assistance for an individual weighing up to 300 pounds. The canes are designed to provide support, increased stability and assistance while walking. The canes are NOT designed to support the total weight of individuals.

Ensure that the snap button fully protrudes through adjustment hole of cane. This ensures that the cane is securely locked in position. BEFORE using the cane, ensure that it is securely locked in place or a fall could result, causing bodily injury and/or damage to the cane.

ALWAYS CONSULT YOUR PHYSICIAN OR THERAPIST TO DETERMINE PROPER ADJUSTMENTS AND USAGE.

### PROPER CANE HEIGHT


For a more comfortable fit, ensure shoe height is the same as shoe style worn most frequently when using cane.

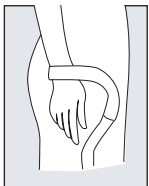

1. Stand as upright as possible on a firm, flat surface and let your arm hang down naturally by your side.
2. Adjust cane so top portion of handle touches inside of wrist where it meets palm.

### HEIGHT ADJUSTMENT

1. Turn the anti-rattle collar knob **COUNTERCLOCKWISE** to loosen.
2. Press **IN** snap button.
3. Slide extension up or down until snap button protrudes through an adjustment hole on handle end of cane.  
**NOTE:** An audible "CLICK" will be heard when snap button securely protrudes through the adjustment hole.
4. Repeat **STEPS 2-3** until cane is at height determined **PROPER CANE HEIGHT** on this instruction sheet.
5. Tighten the anti-rattle collar knob by turning **CLOCKWISE**.

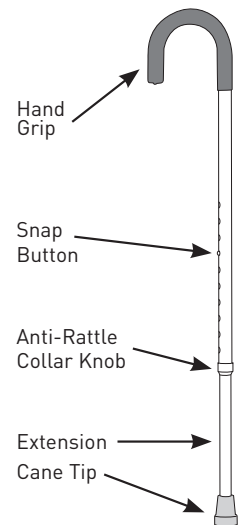

### MAINTENANCE & CARE

1. Ensure snap button releases and re-engages properly.
2. Inspect cane tips for wear and damage.
3. Replace any broken, damaged or worn items immediately.



**WARNING:** This product can expose you to bisphenol A (BPA), which is known to the State of California to cause birth defects or other reproductive harm. For more information, go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).

Technical support, replacement parts and warranty information? Call 1-866-694-5085 General product questions? Call 1-800-777-4908

### Satisfaction Guaranteed

For complete details, please visit [mms.mckesson.com/mckesson-brands](http://mms.mckesson.com/mckesson-brands).

Distributed by McKesson Medical-Surgical Inc.  
Richmond, VA 23233  
PVN B0518  
Made in Taiwan