

McKESSON

PREVENT[®] SG

Tuberculin Syringes with
Safety Hypodermic Needles




GLIDE | 1cc

PERMANENTLY ATTACHED REGULAR WALL NEEDLE


NON-PYROGENIC | STERILE

MFR # 306620


SYMBOL GLOSSARY

	Consult instructions for use
STERILE R	Sterilization using irradiation
	Single use only. Do not reuse.
R_x ONLY	Federal (USA) law restricts this device to sale by or on the order of a physician.
	Consult accompanying documents
LOT	Lot information
EXP	Do not use this product past the indicated expiration as sterility cannot be assured beyond this date.


INSTRUCTIONS FOR USE

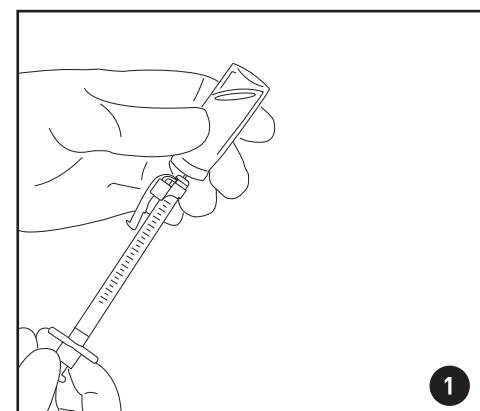


1 Draw up medication in accordance with established protocol.

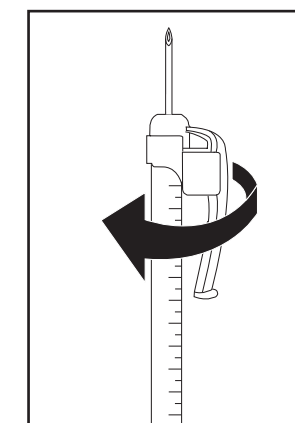


2 Rotate needle protection device for bevel orientation or scale readability.

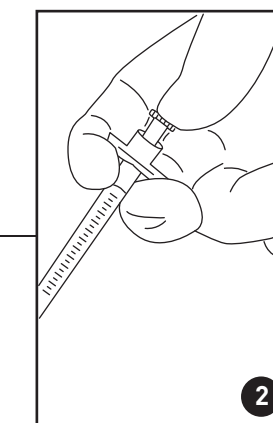

3 Administer medication in accordance with established protocols. For user convenience, the needle protection device can be rotated for ease of injection.

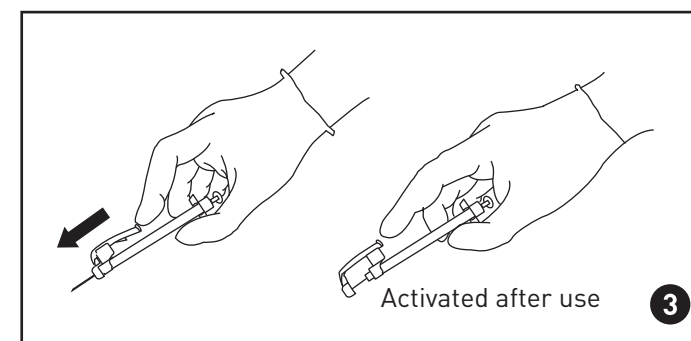

4 Activate safety mechanism immediately after removal from the patient by moving the pushrod completely forward with forefinger until the pushrod is fully extended and the needle tip is covered. Visually confirm that the pushrod has fully advanced and the needle tip is covered. If unable to activate, discard immediately into an approved sharps collector.

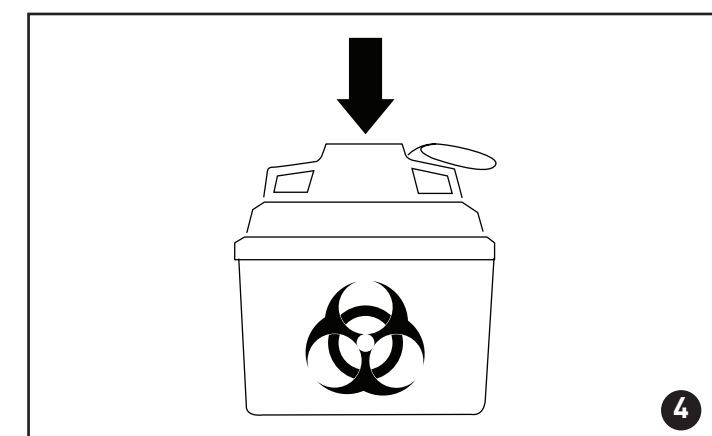

5 Discard after single use in an approved sharps container in accordance with applicable regulations and institutional policy.

CAUTION: Where local and/or institutional procedures permit/require transportation of the filled syringe, use a passive recapping technique to cover the needle before transporting to the point of administration. OSHA standards require that such recapping must be accomplished using a one-handed technique. **DO NOT** hold the needle shield during the recapping process. To help avoid HIV (AIDS), HBV (Hepatitis) and other infectious diseases due to accidental needlesticks, activate the protective mechanism immediately after use. Federal (USA) law restricts this device to sale by or on the order of a physician. Re-use may lead to infection or other illness/injury. Pharmaceutical products prepared into these syringes should be administered as soon as possible.

Use one-handed aseptic technique and activate away from self and others. Non-pyrogenic. Do not use if individual packaging is damaged.

Questions? Call 1-800-777-4908

Satisfaction Guaranteed
For complete details, please visit
mms.mckesson.com/mckesson-brands.

Distributed by
McKesson Medical-Surgical Inc.
Richmond, VA 23233
PVN C0422
1000046642802 500062732
Made in USA


DO NOT REUSE
R_x ONLY
STERILE | R