

# MCKESSON

## Bathtub Grab Bar


14.5 inches

For use with MFR#'s 146-12036.

### INSTALLATION INSTRUCTIONS

1. Clean bathtub thoroughly and wipe dry before applying rail. Contact surfaces must be dry and free of soapy residue to assure secure installation.
2. Loosen knob (A) until rail is wide enough to slip on side of bathtub.
3. Place Safety Rail on bathtub with knob (A) facing outside of tub and tighten until rail feels tight and secure.
4. To test installation of rail, pull forward, back and side to side. If rail slips or moves, make sure surface is free of soapy film and is dry, then replace at desired location and retighten.

Safety Rail often requires retightening from time to time. Check periodically before use and retighten as needed.

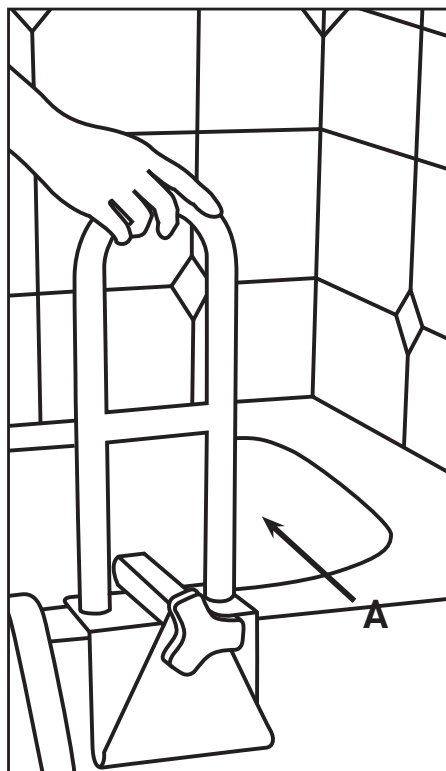

**CAUTION:** Please use extreme care when installing on fiberglass tub. DO NOT OVERTIGHTEN as the clamp may crack the fiberglass.

Technical support, replacement parts and warranty information? Call 1-866-694-5085. General product questions? Call 1-800-777-4908

#### Satisfaction Guaranteed

For complete details, please visit  
[mms.mckesson.com/mckesson-brands](http://mms.mckesson.com/mckesson-brands).

Distributed by McKesson Medical-Surgical Inc.  
Richmond, VA 23233  
PVN B0518  
Made in China