






**MCKESSON**  
**PREVENT® SG**


Syringes with  
 Safety Hypodermic  
 Needles **GLIDE**

LUER SLIP | THIN WALL | NON-PYROGENIC  
 STERILE


**SYMBOL GLOSSARY**

	Consult instructions for use
	Sterilization using irradiation
	Single use only. Do not reuse.
	Federal (USA) law restricts this device to sale by or on the order of a physician.
	Consult accompanying documents
LOT	Lot information
EXP	Do not use this product past the indicated expiration as sterility cannot be assured beyond this date.


**INSTRUCTIONS:**

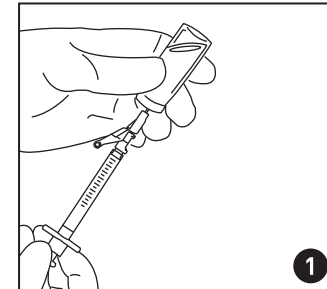


Push firmly when attaching the McKesson Prevent® SG safety needle to the syringe. Draw up medication in accordance with established protocol.

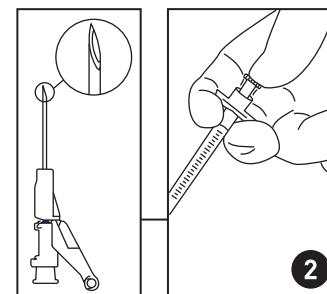

Bevel Up = Lever Arm Up. Administer medication in accordance with established protocols. For user convenience, the needle "bevel up" position is oriented to the lever arm as shown.

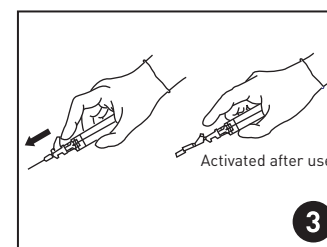

Activate safety mechanism immediately after removal from the patient by moving the pushrod completely forward with forefinger until the pushrod is fully extended and the needle tip is covered. Visually confirm that the pushrod has fully advanced and the needle tip is covered. If unable to activate, discard immediately into an approved sharps collector.

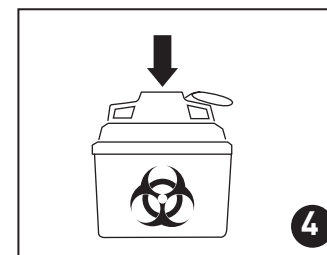

Discard after single use in an approved sharps container in accordance with applicable regulations and institutional policy.

**CAUTION:** Where local and/or institutional procedures permit/require transportation of the filled syringe, use a passive recapping technique to cover the needle before transporting to the point of administration. OSHA standards require that such recapping must be accomplished using a one-handed technique. DO NOT hold the needle shield during the recapping process. To help avoid HIV (AIDS), HBV (Hepatitis) and other infectious diseases due to accidental needlesticks, activate the protective mechanism immediately after use. Federal (USA) law restricts this device to sale by or on the order of a physician. Re-use may lead to infection or other illness/injury. Pharmaceutical products prepared into these syringes should be administered as soon as possible.

Use one-handed aseptic technique and activate away from self and others. Non-pyrogenic. Do not use if individual packaging is damaged.

Questions? Call 1-800-777-4908

**Satisfaction Guaranteed**  
 For complete details, please visit  
[mms.mckesson.com/mckesson-brands](http://mms.mckesson.com/mckesson-brands).

Distributed by  
 McKesson Medical-Surgical Inc.  
 Richmond, VA 23233  
 PVN C0422  
 1000046642703 500050043  
 Made in USA

  
 DO NOT REUSE  
**Rx ONLY**  
