

# Wrist Splint Elastic

For use with MFR #'s 155-79-87072, 155-79-87082, 155-79-87073, 155-79-87083, 155-79-87075, 155-79-87085, 155-79-87077, 155-79-87087, 155-79-87078, 155-79-87088.

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INDICATIONS**

Support to aid carpal tunnel syndrome, immobilization of the wrist.

**PRECAUTIONS**

- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.
- Never wear the support over an open wound.

**DIRECTIONS FOR USE**

1. Place hand in splint with thumb in side slot. The stay should be resting on the palm and in line with the arm.
2. Wrap and secure closures. For supports with D-rings, feed straps through D-rings and secure.
3. The support should be snug but comfortable.

**CLEANING INSTRUCTIONS**



Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY  
 Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

**FOR SINGLE PATIENT USE ONLY.**

Not made with natural rubber latex.

Questions? Call 1-800-777-4908  
 Technical Support? Call 1-800-336-6569

Distributed by McKesson Medical-Surgical Inc.  
 Richmond, VA 23233  
 PVN A0520  
 PN 13-5342 Rev A  
 Made in Mexico

Symbol Definitions	
Symbol	Definition
	Consult instructions for use
	Caution, consult accompanying documents

**NOTICE:** While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability, and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through the use of this product.

# Férula Elástica para Muñeca

Para uso con MFR #'s 155-79-87072, 155-79-87082, 155-79-87073, 155-79-87083, 155-79-87075, 155-79-87085, 155-79-87077, 155-79-87087, 155-79-87078, 155-79-87088.

**ANTES DE UTILIZAR ESTE DISPOSITIVO, LEA POR FAVOR ESTAS INSTRUCCIONES COMPLETA Y DETENIDAMENTE. EL USO CORRECTO ES FUNDAMENTAL PARA EL FUNCIONAMIENTO APROPIADO DE ESTE DISPOSITIVO.**

**INDICACIONES**

Síndrome del túnel del carpo, inmovilización de la muñeca.

**PRECAUCIONES**

- Si experimentara dolor, inflamación, cambios en la sensación o cualquier reacción inusual al usar este producto, consulte a su médico inmediatamente.
- Nunca colocar este soporte encima de una herida abierta.

**LAS INSTRUCCIONES DE USO**

1. Coloque la mano en la férula con la ballena alineada con el brazo y el pulgar descansando en la ranura del costado.
2. Envuelva y fije las correas con un ajuste firme, pero cómodo.
3. La sujeción debe estar bien ajustada, pero confortablemente.



**INSTRUCCIONES DE LIMPIEZA**

El lavado se realiza a mano con jabón suave en agua fría, enjuague exhaustivamente. SECADO AL AIRE. Nota: si no se enjuaga exhaustivamente, los restos de jabón pueden provocar irritación y deteriorar el material.

**PARA USO EN UN SOLO PACIENTE.**

No está hecho con látex de caucho natural.

Distribuido por McKesson Medical-Surgical Inc.  
 Richmond, VA 23233  
 PVN A0520  
 PN 13-5342 Rev A  
 Hecho en Mexico

Definiciones de Símbolos	
Símbolo	Definición
	Consulte las instrucciones de uso
	Precaución: consultar los documentos adjuntos

**AVISO:** Si bien se han hecho todos los esfuerzos posibles con las técnicas más modernas para obtener la compatibilidad máxima de la función, resistencia, durabilidad, y comodidad, no existe garantía alguna de que se evitarán lesiones durante el empleo de este producto.