6.5" (3.25 folded)
165 mm

6.5" (3.25 folded)
165 mm

Presentations

1" x 12" (2.5 cm x 30 cm)	5 ribbons per box
2" x 2" (5 cm x 5 cm)	10 dressings per box
4" x 4" (10 cm x 10 cm)	10 dressings per box
6" x 6" (15 cm x 15 cm)	5 dressings per box

Presentaciones

2.5 cm x 30 cm (1" x 12")	5 bandas por caja
5 cm x 5 cm (2" x 2")	10 apósitos por caja
10 cm x 10 cm (4" x 4")	10 apósitos por caja
15 cm x 15 cm (6" x 6")	5 apósitos por caja

Sorbalgon® Ag

Silver containing calcium alginate dressing
Apósito de alginato de calcio que contiene plata

- Do not re-sterilize
No volver a esterilizar
- Do not reuse
No reutilizar
- See instructions for use
Consulte las instrucciones de uso
- Do not use if package is damaged
No utilizar si el paquete está dañado

STERILE R
Method of sterilization: Radiation
Método de esterilización: Radiación

Temperature limitation
Límite de temperatura

Batch code
Código de lote

Use by
Utilizar antes de

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Keep dry.
Mantener seco.

Keep away from sunlight.
Mantener alejado de la luz del sol.

Rx Only Caution: Federal (USA) Law restricts this device to sale by or on the order of a physician (or properly licensed healthcare practitioner).

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XLIT 2802 Rev. 1 (1118)

165 mm
6.5"

**SORBALGON® AG – Calcium Alginate with Silver Dressing
Instructions for Use**

Description

SORBALGON® AG is a sterile, single-use dressing composed of calcium alginate and silver particles, which absorbs wound exudate and releases silver ions in the dressing. As wound exudate is absorbed, the alginate forms a gel, which assists in maintaining a moist environment to help wound healing and allows intact removal. Total maximum amount of silver particles is 0.14mg/cm2. SORBALGON® AG has a broad spectrum of antibacterial effects and the silver protects the dressing from a broad spectrum of bacteria for up to seven days.

Indications

SORBALGON® AG is indicated for the management of moderate to heavily exuding partial to full thickness wounds, including:

- Postoperative wounds
- Traumatic wounds
- Leg ulcers
- Pressure ulcers
- Diabetic ulcers
- Graft and donor sites

Instructions

1. Debride when necessary and irrigate the wound site in accordance with standard protocols. Remove excess solution from surrounding skin.
2. Choose the appropriate size dressing to cover the wound according to the wound size/area.
3. Cover and secure the SORBALGON® AG with a non-occlusive secondary dressing.
4. Dressing change frequency will depend on wound condition and level of exudate. Initially, it may be necessary to change the dressing every 24 hours. Reapply SORBALGON® AG when the secondary dressing has reached its absorbent capacity or whenever good wound care practice dictates.

Warnings

1. This product is for single patient and for external use only.
2. Frequent or prolonged use of this product may result in discoloration of skin.
3. SORBALGON® AG Calcium Alginate Dressings with Silver are intended for use up to 7 days.
4. Do not use it if the pouch is damaged or opened.
5. Do not use this product on dry wounds without first donating moisture to the wound.
6. Do not use this product during radiation treatment or examinations, such as X-ray, magnetic resonance imaging [MRI], ultrasound or diathermy.

Contraindications

Do not use SORBALGON® AG Calcium Alginate Dressings with Silver on patients who have a silver allergy.

Storage and Disposal

1. Store SORBALGON® AG in a cool and dry place. Avoid light.
2. Disposal should be handled according to local environmental procedures.

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3. Cubra y asegure el apósito SORBALGON® AG con un apósito secundario no oclusivo.
3. Los apósitos de alginato de calcio con plata SORBALGON® AG están diseñados para usarse hasta 7 días.
No use los apósitos de alginato de calcio con plata SORBALGON® AG en pacientes que son alérgicos a la plata.
1. Guarde su SORBALGON® AG en un lugar fresco y seco. Evite la luz.