## Premium Shoulder Immobilizer

For use with MFR #'s 155-79-84100C.

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INDICATIONS

For immobilization aid of the shoulder following surgery, injury, or cast removals.

### PRECAUTIONS

- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

### DIRECTIONS FOR USE

1. Wrap and secure the chestband around the rib cage.
2. Wrap and secure the upper arm strap.
3. Wrap and secure the wrist strap.
4. For immobilizers with shoulder straps, bring strap over uninjured shoulder and secure.
5. For immobilizers with detachable straps, secure straps to chestband prior to application of brace.

### CLEANING INSTRUCTIONS

Hand wash in warm water with mild detergent. Rinse thoroughly and air dry.

### FOR SINGLE PATIENT USE ONLY.

Not made with natural rubber latex.

Questions? Call 1-800-777-4908

Technical Support? Call 1-800-336-6569



Distributed by McKesson Medical-Surgical Inc.

Richmond, VA 23233

PVN A0520

PN 13-5339 Rev A

Made in China

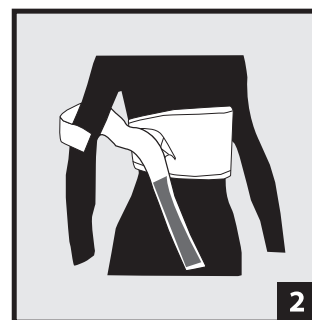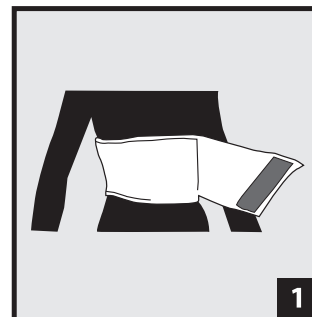Symbol Definitions	
Symbol	Definition
	Consult instructions for use
	Caution, consult accompanying documents

**NOTICE:** While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability, and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through the use of this product.

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Distribuido por McKesson Medical-Surgical Inc.