Universal Wrist Splints

For use with MFR #'s 155-81-87460, 155-81-87470.

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INDICATIONS

Treatment of wrist sprains, strains, and carpal tunnel syndrome (CTS).

CONTRAINDICATIONS:

Phlebitis patients should not wear neoprene products.

PRECAUTIONS

- Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by-products. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

DIRECTIONS FOR USE

- Loosen all straps and position hand in brace. Make sure the palmer stay is positioned comfortably below the palmer crease and aligned with forearm.
- Position adjustable dorsal strap pod to ensure maximum support and proper fit. Stay pod maybe moved proximal or distal for desired controlled range-of-motion.
- Secure strap between thumb and forefinger.
- Use single pull Velstretch® strap to secure body of brace. Straps should be snug but not constrict.

NOTE: Single pull strap should feed through underside of opposing side of brace and D-Ring prior to securing.

CLEANING INSTRUCTIONS

Hand wash liner in warm water using mild soap, rinse thoroughly. AIR DRY.

NOTE: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

FOR SINGLE PATIENT USE ONLY.

Not made with natural rubber latex.

Questions? Call 1-800-777-4908

Technical Support? Call 1-800-336-6569





Distributed by McKesson Medical-Surgical Inc.

Richmond, VA 23233

PVN A0520

PN 13-5512 Rev A

Made in China

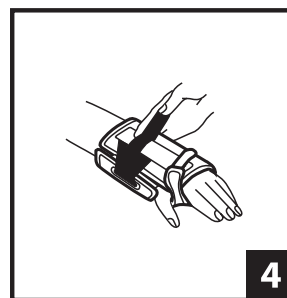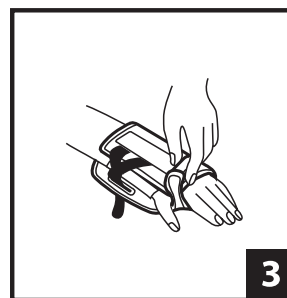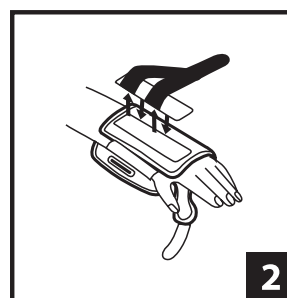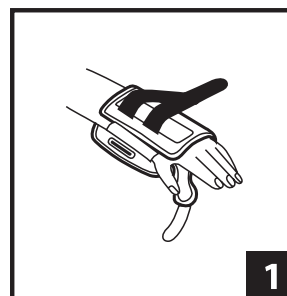Symbol Definitions	
Symbol	Definition
	Consult instructions for use
	Caution, consult accompanying documents

NOTICE: While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through the use of this product.

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Distribuido por McKesson Medical-Surgical Inc.