Abdominal Binder/Rib Belt

For use with MFR #'s 155-79-89070, 155-79-89071, 155-79-89090, 155-79-89091, 155-79-89210, 155-79-89220, 155-81-97150.

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INDICATIONS

Abdominal Binders: To provide compression and support for strains and weakness of the abdominal area. **Rib Belts:** To help provide compression and support for fractures and strains to the rib cage.

PRECAUTIONS

- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

DIRECTIONS FOR USE

Abdominal Binders:

1. Center support at the small of the back. Pull the left side firmly across the front of the body, then repeat for the right side and attach.

Rib Belts:

1. Position belt at rib cage. Wrap left side across the body, then repeat for the right side and secure closures. Deluxe Rib belt will close in the back.
2. Supports and belts should be snug but comfortable.

CLEANING INSTRUCTIONS

Hand wash liner in warm water using mild soap, rinse thoroughly. AIR DRY.

NOTE: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

Rib belts are most effective when worn over bare skin or undergarments.

FOR SINGLE PATIENT USE ONLY.

Not made with natural rubber latex.

Questions? Call 1-800-777-4908

Technical Support? Call 1-800-336-6569

Distributed by McKesson Medical-Surgical Inc.

Richmond, VA 23233

PVN A0520

PN 13-5334 Rev A

ABD Binders - Made in China

Rib Belt - Made in Mexico

Symbol Definitions	
Symbol	Definition
	Consult instructions for use
	Caution, consult accompanying documents

NOTICE: While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability, and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through the use of this product.

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Distribuido por McKesson Medical-Surgical Inc.