

MCKESSON

Fluid-Resistant Surgical Masks

CLASSIC STYLE WITH TIES | EYE SHIELD | ANTI-FOG


For use with MFR numbers: 91-1600, 91-2600

25


PER BOX




SINGLE USE ONLY

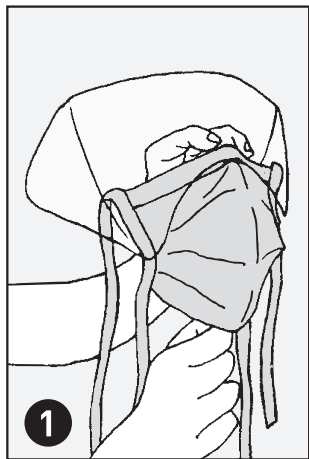


1
Open the pleats and pre-bend the nose piece in the middle.

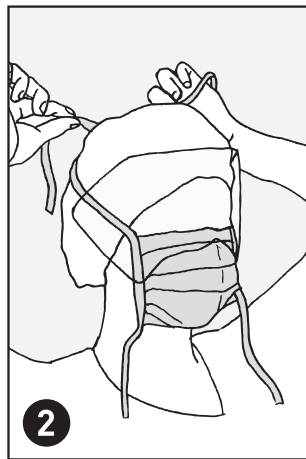

2
Place the mask over your nose and tie the top ties at the crown of your head.

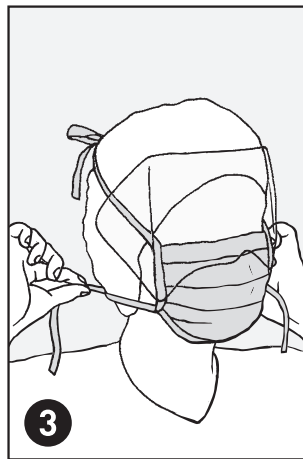

3
Tie the bottom ties behind your neck. There should be no large gaps at the sides of the mask.

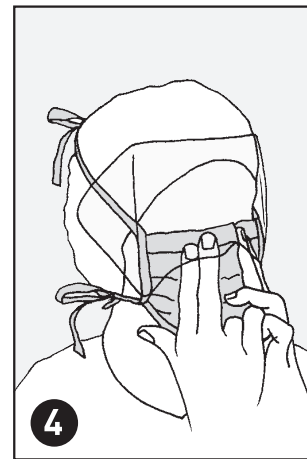

4
Slide your fingers underneath the visor to tighten the nose piece.

ASTM F2100-11 - Level 3 Barrier

Fluid Resistance: ASTM F1862 at 160 mm Hg

BFE: $\geq 99\%$

PFE: $\geq 99\%$

Delta-P: ≤ 2.7

Flammability: Class 1

Satisfaction Guaranteed

For complete details, please visit
mms.mckesson.com/mckesson-brands.

Distributed by McKesson Medical-Surgical Inc.

Richmond, VA 23233

PVN A0823

Made in China