Instructions for use

General description of the device

The TwoPress 2* compression system set is a combination of two non-woven bandages. One of the bandages is made of polypropylene and elastane and is brown in color, the other one is made of polyester and elastan and is white in color. The brown one has a cohesive layer on both sides, the white one has a cohesive layer on one side. The non-active and non-sterile device is available as a kit. Both bandages are 10 cm in width. The indicator is an application guide for therapeutically effective compression.

Intended purpose

The non-sterile 2 component compression system TwoPress 2 can be used for long-term compression therapy:

- treatment of chronic venous insufficiency (CVI) with and without venous leg ulcers (VLU)
- thrombosis / thrombophlebitis
- chronic oedemata of other origin (ankle brachial pressure index -ABPI 0.8-1.3)
- lymph oedemata

It is for single-use and can be used by healthcare professionals in clinical and home environments.

Composition

Padding bandage: 92% polyester + 8% elastane, with a cohesive coating on one side. Compression bandage: 85% polypropylene + 15% elastane, with a cohesive coating on both sides.

Indication

TwoPress 2 is indicated for the treatment of chronic venous insufficiency (CVI) with and without ulcer cruris venosum (UCV), thrombophlebitis, varicosis and chronic oedemata of other origins (ankle brachial pressure index - ABPI 0.8 - 1.3).

Intended users

TwoPress 2 is designed to be used by healthcare professionals who have received appropriate training. The compression bandage system can be used for the treatment of: varicose veins, Thrombosis / thrombophlebitis, post-thrombotic syndrome, thrombophylaxis chronic venous insufficiency (CVI), venous insufficiency, angiodysplasia and other oedemata.

Contraindications

TwoPress 2 must not be used in case of:

- advanced peripheral arterial occlusive disease (PAOD)
- decompensated heart failure (NYHA II +IV)
- phlegmasia coerulea dolens
- in case of ankle-brachial pressure index (ABPI) > 1.3 and ≤ 0.5
- septic phlebitis

Precautions

Special precautionary measures are required in the event of:

- pronounced sensibility disorders of the extremities (e.g. in the case of advanced diabetic peripheral neuropathy)
- pronounced weeping dermatoses
- incompatibility with compression bandages material
- primary chronic polyarthritis

The application technique may have to be adapted to the patient's anatomical and pathophysiological characteristics (e.g. as regards stretch, padding, etc.).

If the foot pulse is not palpable, a Doppler ultrasound should be carried out to determine the ABPI prior to starting therapy.

If an inelastic system is created by close to full stretch application, high ankle pressures should be closely monitored by a physician.

Application

Please use this system as a set and in the correct order. When applying the two bandages, make sure that the patient's foot is at an angle of 90° relative to the lower leg (neutral 0 position) ❶. Examine the leg before application to identify bony prominences and potential pressure points and pad them with the padding bandage. Start applying the padding bandage (1) with the non-cohesive side against the skin proximal to the base of the toes. After two turns for fixation (plantar, from medial to lateral, to support pronation), wrap the bandage across the back of the foot and enclose the heel in figures of 8. Wrap the foot without stretching the bandage ❷❸.

On the lower leg, wrap the padding bandage around the leg using a spiral technique with an overlap of 50%, keeping the bandage at full stretch ❹. The stretch may have to be adapted to the patient's anatomical and pathophysiological characteristics. Tear off any excess bandage material beneath the telerostas tibiae and about 2 cm below the fibular head ❺. Additional fixation is not necessary; tape can be used if required.

As with the padding bandage, start applying the compression bandage (2) proximal to the base of the toes. After two turns for fixation, wrap the bandage across the back of the foot and enclose the heel in figures of 8. Wrap the foot without stretching the bandage and do not yet follow the printed indicator ❻.

Wrap the cohesive compression bandage spirally around the leg with an overlap of 50%. To achieve the optimal pressure, stretch the bandage until the indicator printed on it has reached the length of a regular hexagon with all sides being the same length ❼. The overlap must be 50%. For better orientation, use the centre line of the hexagon ❽. Check the elongation using the template found on the flap of the packaging. Press out the perforated hexagon template and use it to check the correct elongation of the 2nd layer. If the indicator is correctly stretched, you will see the printed indicator at the edge of the template. The stretch may have to be adjusted to the patient's anatomical and pathophysiological characteristics.

Tear off any excess bandage material beneath the telerostas tibiae and about 2 cm below the fibular head. Carefully press the bandage onto the entire leg. This will ensure optimal cohesion ❹❽.

Additional fixation is not necessary; tape can be used if required.

Further remarks

TwoPress 2 is indicated for use by medical professionals who have received appropriate training. Before applying TwoPress 2, clean the wound and cover it with a suitable wound dressing. If required, excess padding bandage material may be used for additional padding of bony prominences and tissue recesses (e.g. shinbone, Bissgaard's backdrop). If additional fixation is required, use tape but no bandage clips.

After application, the compression bandage must be checked (e.g. circulation in the patient's toes, pain reported by the patient) in order to be certain that the pressure of the bandage is not too high.

TwoPress 2 is intended for single use only.

Both bandages can be torn by hand.

Both bandages are made without natural rubber latex.

The applied bandage can be removed either by unwrapping, or by carefully cutting it with bandage scissors.

The bandage can be worn day and night for up to 7 days. The specific application period must be determined individually based on the particular patient's wound / skin/oedema situation. The application technique must be followed in order to achieve the therapeutically recommended compression pressure. If the bandage becomes undone or slips, the bandage should be removed and a new system applied in consultation with your medical staff.

If at any time during therapy with TwoPress 2 the patient develops severe pain, numbness, tingling or noticeable discolouration of the toes, a medical professional must be contacted immediately and the bandage must be removed.

When applying TwoPress 2, make sure that the tension is even as the hexagon indicator reaches the shape of a regular hexagon ❹ and the bandages are straight in order to achieve uniform pressure distribution and avoid constrictions.

During therapy with TwoPress 2, the patient should be encouraged to move.

An important element of compression therapy is the active muscle pump, which acts as counterpart to the working pressure of the compression bandage.

Incident reporting

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request.

Product disposal

In order to minimize the risk of potential infection hazards, or environmental pollution, disposable components of the medical device should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards. Dispose of the medical device with regular hospital waste.

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* © MEDICE-Iserlohn
PAUL HARTMANN AG
HARTMANN USA, Inc.