


McKESSON


PREVENT[®]

Instructions For Use

McKesson Prevent[®] Safety Hypodermic Needles and Syringes

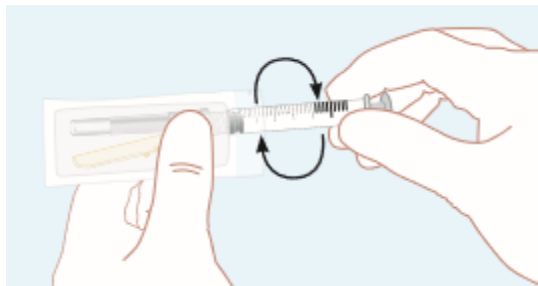


Step 1. Tighten the syringe to the needle using aseptic technique. Grip the base of the needle, not the safety sheath, and turn the syringe clockwise.

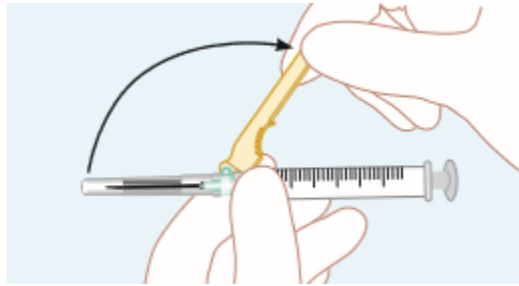


Step 2. Move the safety sheath away from the needle and toward the syringe barrel to the angle shown, prior to removing the needle cap. Perform injection procedure according to established technique.



Note: The needle's "bevel up" position is oriented to the position of the safety sheath.

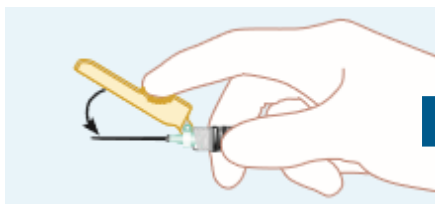

FOREFINGER

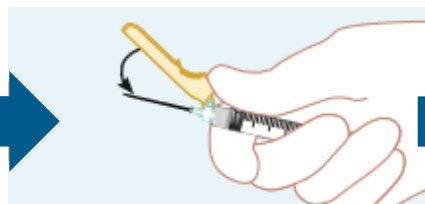


THUMB

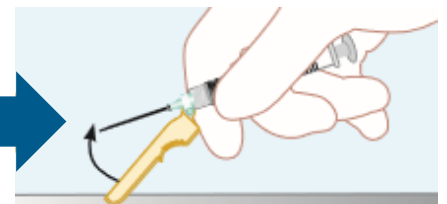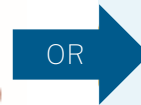

HARD SURFACE

Step 3. Activation: After the injection procedure, use a one-handed technique to activate the safety mechanism using any of the three (3) methods illustrated. (Activation is verified by an audible and/or tactile "click," and can be visually confirmed.)

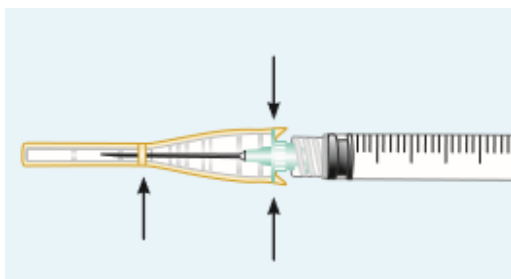

SAFETY LOCK CONFIRMATION

Step 4. Visually confirm that the safety sheath is fully engaged.

Warnings:

- Handle with care to avoid needlesticks
- Do not replace original needle cover after use
- Use once and discard immediately in accordance with local and safety standards

Cautions:

- If needle is bent or damaged, no attempt should be made to straighten needle or use the product
- Do not attempt to deactivate the safety device by forcing the needle out of the safety sheath
- For single use only. Do not reuse. Do not resterilize.

Precautions:

- Keep hands behind needle at all times during use and disposal
- Observe Universal Precautions on ALL patients
- Do not use if the unit package or product has been damaged or contaminated
- Do not store at extreme temperature and humidity. Avoid direct sunlight.

Step 5. Dispose of used needles and materials following the policies and procedures of your facility as well as federal and local regulations for sharps disposal.

■ McKesson Brands' 100% Satisfaction Guarantee

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit within 12 months from the purchase date. McKesson Brands' satisfaction guarantee is limited to products purchased directly from McKesson. If a McKesson Brands product is not purchased directly from McKesson, any return request or request for refund must be directed to the selling party.