Ankle Brace Stirrup

For use with MFR #'s 155-81-97877.

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INDICATIONS

Acute injuries of the ankle.

PRECAUTIONS

- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

DIRECTIONS FOR USE

1. Open ankle stirrup so gel and closed cell foam are facing upward.
2. Position the heel evenly on the center pad. Wrap lower hook and loop closure strap around ankle for alignment. Center side shells along ankle and leg. Wrap upper strap and adjust if necessary.
3. Apply pressure to the sides of the therapeutic gel bag to allow even flow of pressure within the plastic shell. Tighten lower and upper straps for a secure fit and patient comfort. Straps may be trimmed if necessary.
4. Adjust vertical side straps, placing both hands on straps, pulling upward until desired fit. Secure vertical side straps with hook and loop closure.

NOTE: The ankle brace with gel should be worn over a piece of stockinette or thin cotton sock.

COLD THERAPY

- Remove therapeutic gel bag from ankle support system and place in freezer for 2 hours.
- Place gel bag in ankle support and follow fitting instructions.

NOTE: Absorbent sock should be worn before ankle support is fitted.

CLEANING INSTRUCTIONS

Outer shell and gel bag may be hand washed with warm water and mild detergent. AIR DRY. Freeze only therapeutic bag. DO NOT place entire system in freezer.

FOR SINGLE PATIENT USE ONLY.

Not made with natural rubber latex.

Questions? Call 1-800-777-4908

Technical Support? Call 1-800-336-6569

Distributed by McKesson Medical-Surgical Inc.

Richmond, VA 23233





PVN A0520

PN13-5527 Rev A

Made in Mexico

Symbol Definitions	
Symbol	Definition
	Consult instructions for use
	Caution, consult accompanying documents

NOTICE: While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through the use of this product.

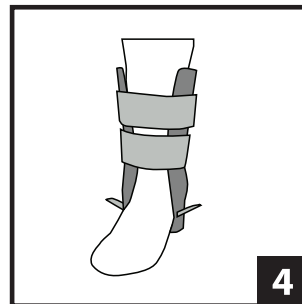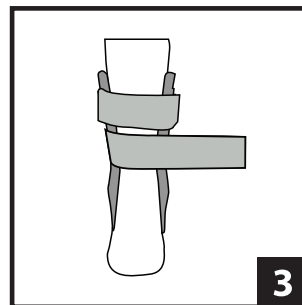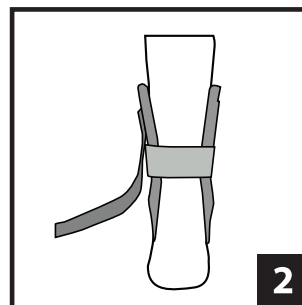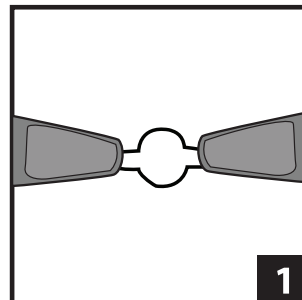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Distribuido por McKesson Medical-Surgical Inc.