MCKESSON

Ankle Brace, Figure Eight

For use with MFR #'s 155-81-97042, 155-81-97043, 155-81-97045
155-81-97047, 155-81-97048.

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INDICATIONS

Acute ankle sprains • Chronic ankle sprains • Prophylactic use
Ankle brace should be worn with an athletic sock.

DIRECTIONS FOR USE

1. Before application, loosen straps and laces. The ankle must be at a 90° angle when applying the support. While seated at the edge of a chair, place foot in ankle support. Make sure neoprene tongue of support is on the top of the foot area prior to lacing-up. Lace up the support like a shoe.
2. For application to the right foot, turn heel out slightly. For left foot, turn heel in slightly. Starting with strap "A," wrap over forefoot and then under the arch area of the foot. Secure with tension to pressure-sensitive closures. Trim strap "A" if necessary.
3. For right foot, rotate heel slightly inward. For left foot, rotate heel slightly outward. Apply strap "B" over forefoot and then under arch area of foot. Secure with tension to pressure-sensitive closure. Trim strap "B" if necessary.
4. Wrap circumferential strap around ankle over straps "A" and "B" to secure to itself. NOTE: Ankle support should be snug but not impair circulation.

CLEANING INSTRUCTIONS

Hand wash with mild soap in lukewarm water. Rinse thoroughly. AIR DRY.
NOTE: If not rinsed thoroughly, residual soap may cause skin irritation or deteriorate the material. Never wear the support over an open wound.

FOR SINGLE PATIENT USE ONLY.





Not made with natural rubber latex.

Questions? Call 1-800-777-4908
Technical Support? Call 1-800-336-6569

Distributed by McKesson Medical-Surgical Inc.
Richmond, VA 23233
PVN A0520
PN 13-5500 Rev A
Made in China

Symbol Definitions	
Symbol	Definition
	Consult instructions for use
	Caution, consult accompanying documents

NOTICE: While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through the use of this product.

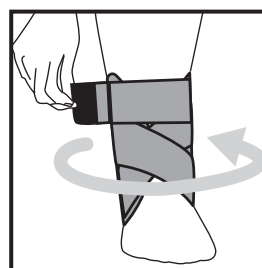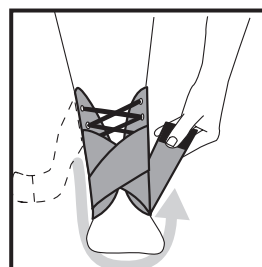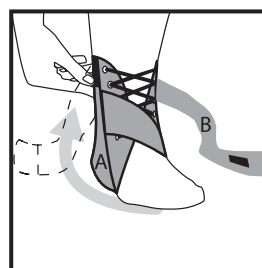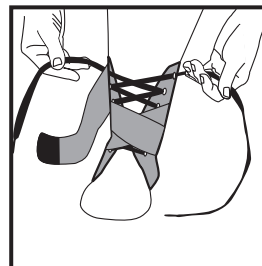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Distribuido por McKesson Medical-Surgical Inc.