

McKESSON

Portable Transport Stretcher

14 HANDLES | FLUID RESISTANT

Transports, transfers, or rescues patients from areas that are inaccessible to standard stretchers. Provides a total weight bearing capacity of 1000 lbs (453.6 kg).


40 in x 80 in

(101.6 cm x 203.2 cm)


1

PER PACK

PROPER GRASPING

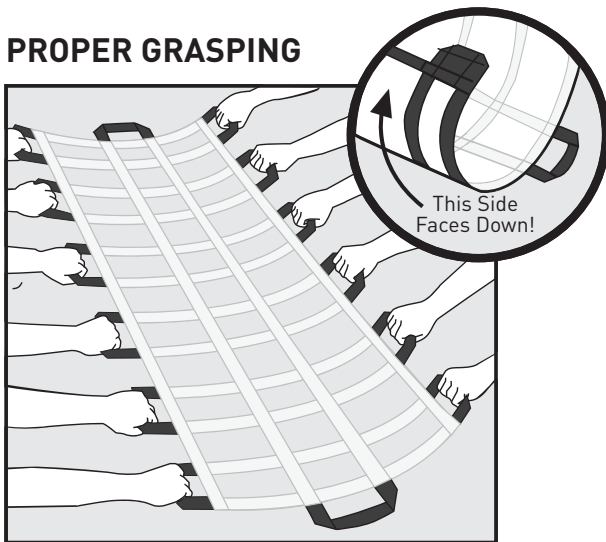

IMPROPER GRASPING

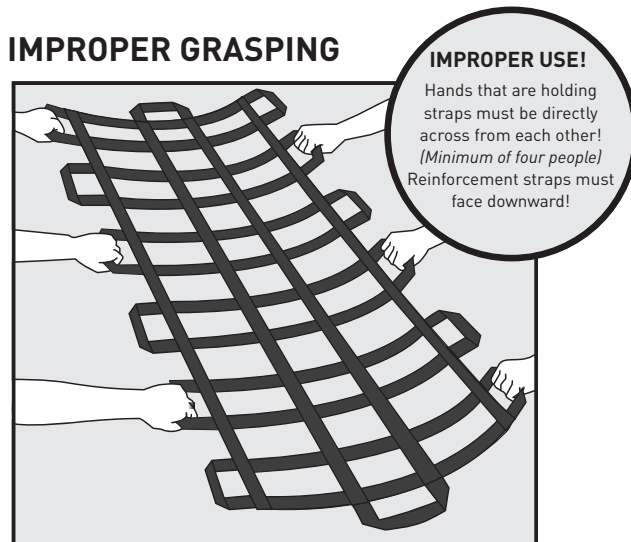

The McKesson Portable Transport Stretcher is for use by professional staff for lifting and transporting/transferring patients. The McKesson Portable Transport Stretcher is not designed for use when there is a need for spinal stabilization or traction of the patient being transported. The unit must be placed so handle support straps are to the side opposite the patient. It is recommended to use at least four (4) people to provide safe transport/transfer, thus reducing the possibility of accident or injury. Hands that are holding straps must be directly across from each other – do not “cross-handle.” Only use handles for lifting. Use care to ensure grip is secure prior to lifting.

HOW TO CLEAN

Clean soiled Portable Transport Stretcher with a damp cloth. Soap, detergent or disinfectant can be used.

WARNING: Improper handling or use of a damaged Portable Transport Stretcher can cause death or serious injury. Follow all safety instructions.

CAUTION: Intended for limited use. Do not use if punctured, torn, frayed, or excessively worn. Do not machine wash or dry. Avoid contact with sharp objects. Do not store in contact with heat source greater 200°F. Avoid dragging over rough surfaces. Do not attach to mechanical lifts.

Not made with natural rubber latex.

Questions? Call 1-800-777-4908

Satisfaction Guaranteed

For complete details, please visit
mms.mckesson.com/mckesson-brands.

Distributed by McKesson Medical-Surgical Inc.

Richmond, VA 23233

PVN A1219

Made in China

MFR # **143-51926**



6 12479 24906 6