ThreePress®

Compression Bandaging System

Indications

The THREEPRESS® Compression Bandaging System is indicated for the management of venous leg ulcers and related conditions. THREEPRESS® may be used on patients with an ankle circumference 18 cm or larger (padded). Compression data indicates that the THREEPRESS® system provides sustained compression in the 30-40 mmHg range for up to seven days at the ankle when applied according to package insert directions.

Contra-Indications

THREEPRESS® is not indicated for use on diabetic patients with small vessel disease or on patients with an ankle brachial index (ABPI) of less than 0.8. Doppler ultrasound is recommended to rule out arterial disease.

Note to Patients

If during use of this product, it becomes wet, slips or gaps, becomes uncomfortable, creates irritation or you observe uncharacteristic color or odor from your foot, consult your clinician immediately.

Application Instructions

Before applying the first bandage:

- Check the ABPI (Ankle Brachial Pressure Index) to determine if arterial disease is present. If the ABPI is less than 0.8 do not apply the System.
- Confirm ankle circumference is greater than 18 cm. (Ankle circumference may change as edema is reduced.)
- Apply an appropriate non-adhesive primary dressing to cleansed wound before applying THREEPRESS®.

Layer One - Padding Bandage: Wrap the padding/absorption layer with the fluffy cotton side against the skin. Use a spiral technique at 50% overlap from the base of the toes to just below the knee. Stretch this layer around bony prominence, such as the heel, to increase conformability. Cut off any excess bandage and use it over the wound area to absorb drainage. Secure the end of the padding layer with one of the tape strips provided. **When applying a compression system to the leg, it is important to enclose the heel with all layers of the System.**

Layer Two - Compression Bandage: Wrap the compression bandage from the base of the toes to the ankle using the spiral technique. At the ankle, switch to using the figure of eight wrapping technique at 50% extension and 50% overlap starting at the ankle to just below the knee. To assist in extending this bandage at 50% the rectangular pattern on the bandage will form a square pattern at 50% extension. Cut off any excess bandage once you have wrapped to the knee. Secure the end of Layer Two with one of the tape strips provided.

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Layer Three - Cohesive Compression Bandage:

Wrap the cohesive compression bandage from the base of the toes to just below the knee using the spiral technique at 50% extension and 50% overlap. Cut off any excess bandage to avoid a tourniquet effect once you have wrapped to the knee.

Removal: THREEPRESS® should be cut off by a clinician with specially designed bandage scissors. After removal, clean and assess the wound, chart the progress of healing and continue treatment if necessary. Dispose of the contaminated dressing and wraps using the plastic bag in which THREEPRESS® is packaged.

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