


Autoclave Indicator Tapes

For use with MFR #'s 73-BT036, 73-GT036, 73-ST036, 73-ST048.


INSTRUCTIONS FOR USE

1. Open the plastic wrapper and place in a compatible tape dispenser.
2. Unwind a section of the tape approximately 20-30 cm (8-12 inches) long.
3. Place one end of the indicator on the package and apply finger pressure across the length of the tape to secure the pack. See illustrations below.
4. Do not stretch the tape by pulling on it, which may cause bunching of the material.
5. Repeat steps 2 and 3, utilizing two strips of tape to secure the package/tray.
6. After sterilization allow pack / wrapped tray to cool 30 minutes prior to breaking the seal. The chemical indicator lines shall turn dark brown/black indicating a sterilization process occurred.
7. Do not remove chemical indicator tape in operating room because linting may occur.
8. Remove sterilization tape from linens and untreated woven fabrics, to prevent adhesive from gumming or sticking.


Sequential Double-Wrapping: envelope fold

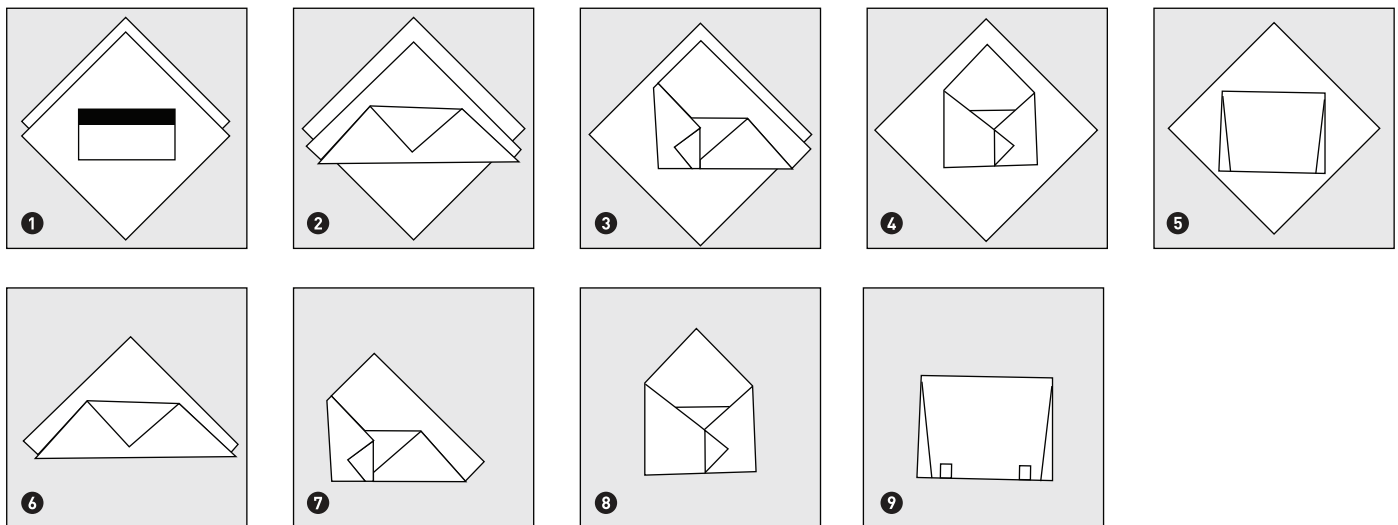


Simultaneous Double-Wrapping: envelope fold

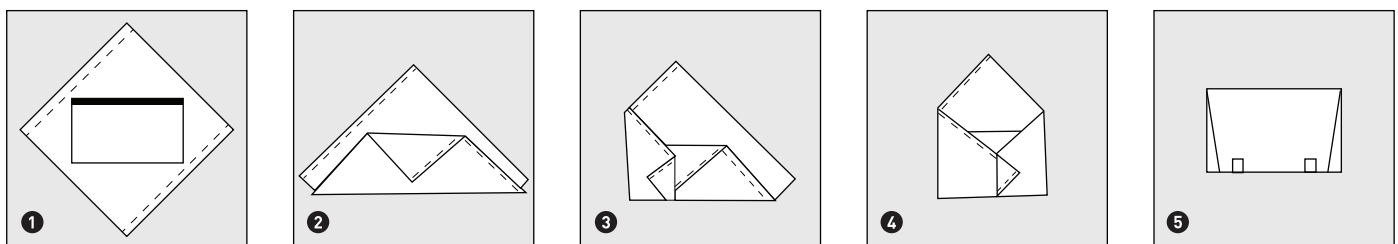

Sequential Double-Wrapping: square fold

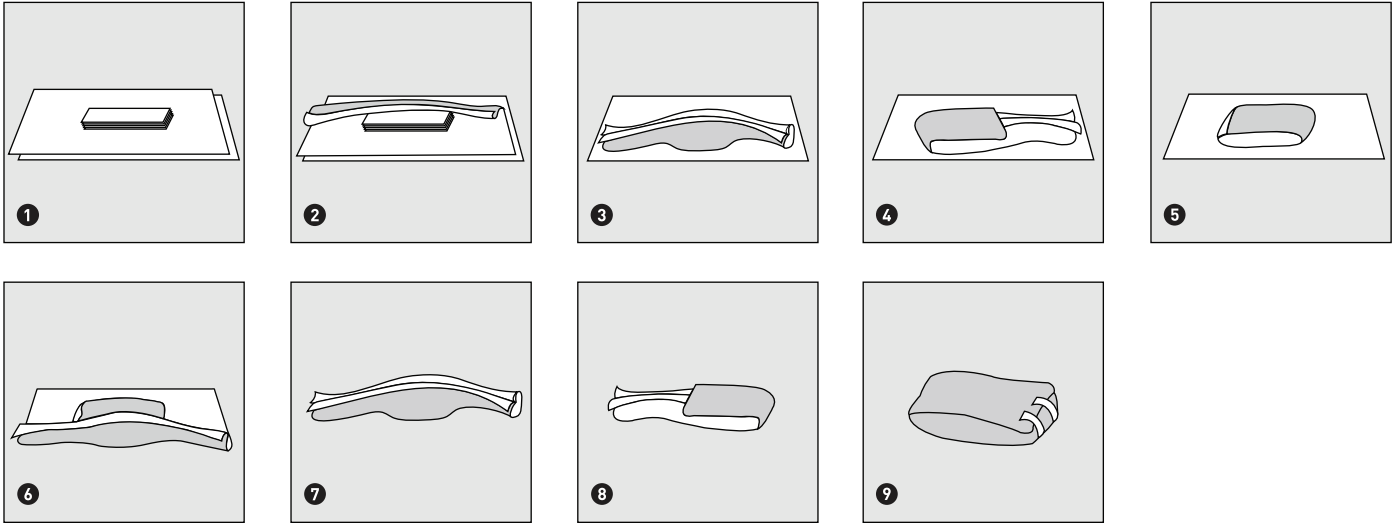

Simultaneous Double-Wrapping: square fold

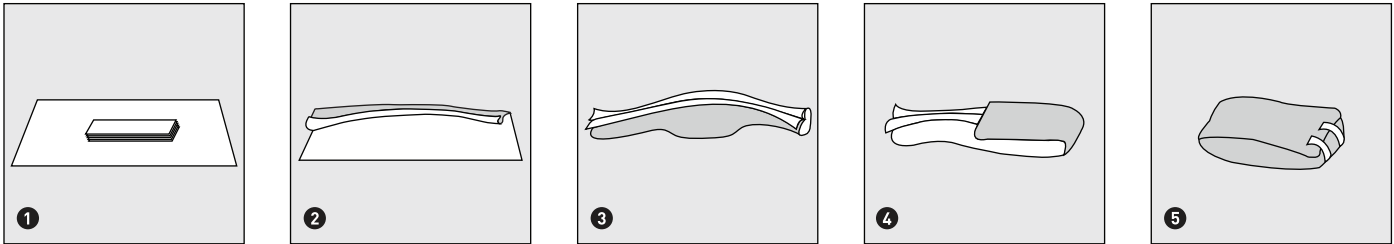

Two single-layer
wrappers or one bonded
double-layer wrapper

CAUTION: THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS.

Store in a cool, dry place. Store at controlled room temperature as defined by the United States Pharmacopeia. Controlled room temperature is thermostatically controlled between 68-77 °F (20 -25 °C), with a mean kinetic temperature of not more than 25 °C, with excursions to 59-86 °F (15 °C-30 °C) allowed. Reference the USP for the complete definition.

Questions? Call 1-800-777-4908

■ Satisfaction Guaranteed

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit.

Distributed By McKesson Medical-Surgical Inc.

Richmond, VA 23233

Rev. 00 06/15

Made in Canada