



OPTILOCK®

NON-ADHERENT SUPER ABSORBENT WOUND DRESSING



PREPARATION:

Step 1: Cleanse wound environment using an appropriate wound cleanser.

Step 2: Ensure skin surrounding wound is dry and apply appropriate skin prep, as preferred.

APPLICATION:



1. Remove OptiLock® from the package.



2. Place white side of dressing on the wound. Use a large enough dressing to cover the wound and surrounding skin.



3. Secure dressing in place with an appropriate secondary dressing.

See application video at www.medline.com/awcvideos

© 2018 Medline Industries, Inc. Medline is a registered trademark of Medline Industries, Inc. MKT1895695 / e18048

ABSORBENT POLYMER