

Blockade™ Reusable Gowns

Superior protection combined with comfort and functionality.



**Protection
you can
count on
every time**

Blockade™ covers you with confidence

Blockade is the do-it-all gown. The fabric is made of 99% polyester with 1% carbon to help dissipate static electricity. Its high density construction resists fluids while maintaining comfort and functionality no matter the procedure—all while being extremely durable, environmentally friendly¹ and cost effective.




Durable



Industrial laundry
75 wash cycles




Static resistant

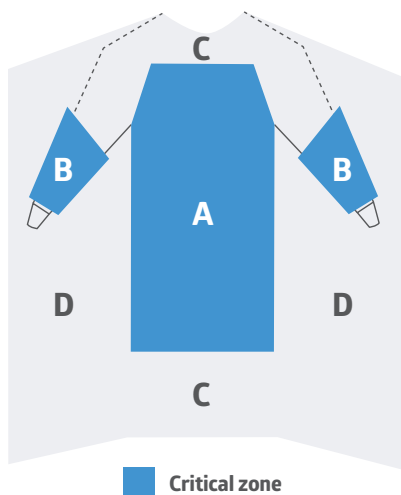

Additional Information

Low-linting, not made with PFOS/PFOA

Latex free:	Not made of natural rubber latex
Sterility:	Ships non-sterile
Traceability:	Lot number on each case of product
Expiration date:	No expiration date
Storage conditions:	General warehouse conditions
Fabric weight:	2.6 OSY

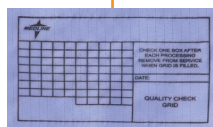

Gowns feature a quality control grid that allows you to track usage

1-ply Blockade gown

This basic style maintains all the features of the Blockade gown and is ideal for a variety of common medical procedures and tasks.

Color-coded collar indicates gown size

Ceil Blue	Large
White	X-Large
Green	XX-Large



1-ply Blockade fabric (entire gown)


MDT012086L



Ties (or snaps) at the neck



Ties at mid-back

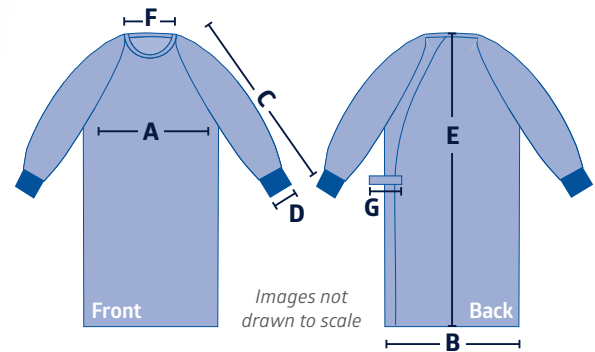

Gown Measurements

	Snaps at the neck, ties at mid-back			Ties at the neck, ties at mid-back	
	SKU: MDT012080L	MDT012080XL	MDT012080XXL	MDT012086L	MDT012086XL
Size:	Size L	Size XL	Size XXL	Size L	Size XL
Gown Color:	Ceil Blue	Ceil Blue	Ceil Blue	Ceil Blue	Ceil Blue
Collar Color:	Ceil Blue	White	Green	Ceil Blue	White
A Chest	23"	25"	27"	22.875"	24.875"
B Sweep	62.25"	68.75"	73"	63"	69.5"
C Sleeve Length	32"	32.5"	33"	32.375"	33.375"
D Sleeve Opening	10"	10"	10"	11.75"	12.75"
E CBL	47.25"	47.5"	47.25"	47.625"	47.5"
F Collar	27.25"	29"	31.25"	28.125"	29.75"
G Tie	21.5"	21.5"	21.5"	21.5"	21.5"

1-ply Blockade on front and AngelStat back

This value-driven style features our 1-ply Blockade fabric on the front of the gown and our lighter-weight AngelStat poly/cotton blended fabric on the back for enhanced breathability and comfort.

Color-coded collar indicates gown size

Ceil Blue	Large
White	X-Large
Green	XX-Large



1-ply Blockade fabric on front


MDT012091L



Snaps (or ties) at neck



AngelStat fabric on back
Ties (or snaps) at mid-back

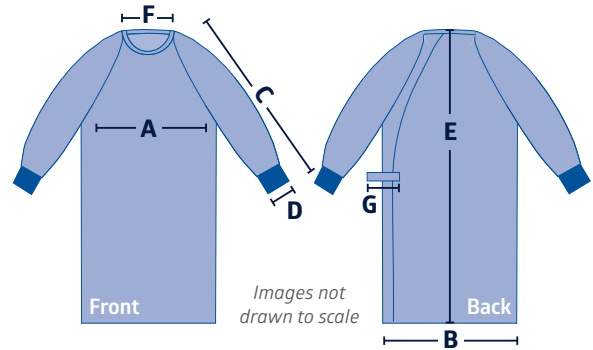

Gown Measurements

	Ties at the neck, ties at mid-back		Snaps at the neck, ties at mid-back			Snaps at the neck, snaps at mid-back		
	SKU: MDT012090L	MDT012090XL	MDT012091L	MDT012091XL	MDT012091XXL	MDT012092L	MDT012092XL	MDT012092XXL
Size:	Size L	Size XL	Size L	Size XL	Size XXL	Size L	Size XL	Size XXL
Gown Color:	Ceil Blue	Ceil Blue	Ceil Blue	Ceil Blue	Ceil Blue	Ceil Blue	Ceil Blue	Ceil Blue
Collar Color:	Ceil Blue	White	Ceil Blue	White	Green	Ceil Blue	White	Green
A Chest	28"	30"	28"	30"	32"	28.5"	30.5"	32.5"
B Sweep	62.125"	67"	62.125"	67"	72.75"	62.5"	67.5"	72.5"
C Sleeve Length	32"	32.75"	32"	32.5"	33.25"	31.5"	32.125"	33.5"
D Sleeve Opening	9.25"	9.25"	12"	12.5"	13"	10"	11"	12.5"
E CBL	50.25"	50.5"	50.25"	50.5"	50.5"	50"	50.25"	51"
F Collar	21.75"	23"	21.75"	23"	24.75"	22"	23"	25"
G Tie	17.5"	17.5"	17.5"	17.5"	17.5"	4.5"	4.5"	4.5"

2-ply Blockade gown

Engineered with 2-ply Blockade fabric in critical zones for added protection, this gown comes in a variety of colors with ties at neck and mid-back for an optimal and secure fit.

Color-coded collar indicates gown size

Yellow	Small
Jade Green	Medium
Ceil Blue	Large
White	X-Large

2-ply Blockade fabric on front and 3/4 sleeves




MDT012083L



Ties (or snaps) at neck



Ties (or snaps) at mid-back

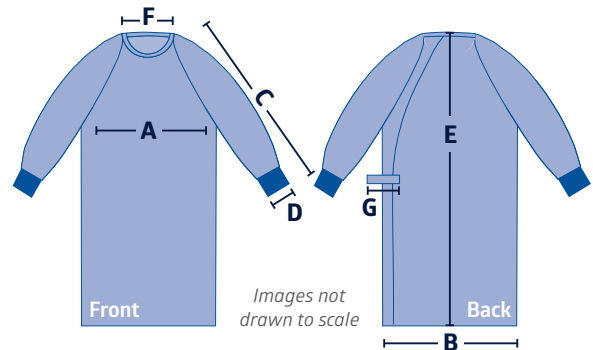

Gown Measurements

Ties at the neck, ties at mid-back

	SKU: MDT012083S	MDT012083M	MDT012083L	MDT012083XL
Size:	Size S	Size M	Size L	Size XL
Gown Color:	Ceil Blue	Ceil Blue	Ceil Blue	Ceil Blue
Collar Color:	Yellow	Jade Green	Ceil Blue	White
A Chest	19.5"	21.25"	23.75"	27"
B Sweep	52"	57.25"	62"	69.5"
C Sleeve Length	30"	31"	31.5"	32.75"
D Sleeve Opening	9"	10"	10"	11"
E CBL	46"	47.25"	47.25"	47.25"
F Collar	24.5"	25.875"	27.25"	29"
G Tie	21"	21"	21"	21"

2-ply Blockade gown (continued)




Ceil Blue	Large
White	X-Large
Mint Green	XX-Large



Ties (or snaps) at neck



Ties (or snaps) at mid-back

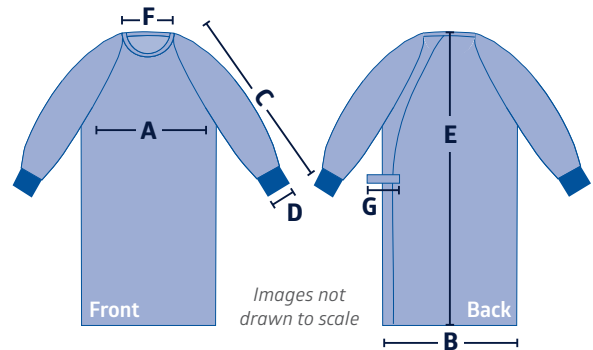

Gown Measurements

	Ties at the neck, ties at mid-back		Snaps at the neck, ties at mid-back			Snaps at the neck, snaps at mid-back	
	SKU: MDT012081L	MDT012081XL	MDT012085L	MDT012085XL	MDT012085XXL	MDT012084L	MDT012084XL
Size:	Size L	Size XL	Size L	Size XL	Size XXL	Size L	Size XL
Gown Color:	Misty Green	Misty Green	Ceil Blue	Ceil Blue	Ceil Blue	Ceil Blue	Ceil Blue
Collar Color:	Ceil Blue	White	Ceil Blue	White	Mint Green	Ceil Blue	White
A Chest	22.875"	24.875"	27"	29.5"	31.75"	22.875"	24.875"
B Sweep	63"	70.5"	63"	69"	73.5"	63"	68"
C Sleeve Length	32.375"	32.75"	32.25"	32.75"	33.25"	32.375"	32.875"
D Sleeve Opening	10.875"	11.875"	10"	12"	13"	11.625"	12.625"
E CBL	47.625"	47.5"	47.5"	47.5"	47.5"	47.75"	47.625"
F Collar	27.625"	29.25"	28"	29"	30.5"	27.5"	29"
G Tie	21"	21"	21"	21"	21"	6"	6"

Laundering recommendations for blockade gowns

Normal wash

Operation	Temperature	Wash time
Suds	72°C/162°F	40 min.
Rinse	60°C/140°F	5 min.
Rinse	60°C/140°F	10 min.
Wet Tumble	–	5 min.

DO NOT use softener.
For best results use batch washer.

Bleach

Do not bleach—may cause damage to barrier properties

Normal

Tumble dry: 66°C +/-5°C | 150°F +/-10°F

MAX temperature :71°C/160°F

With final 10 minute cool-down until fabric is dry

Iron

Do not flat-work iron

Prevac Steam Sterilization

Product provided non-sterile

To sterilize product: 134°C/274°F

4 minute exposure

20 minute exhaust

All products:

Must be laundered prior to use

Only wash with like items

Only wash with like colors

AAMI Life Cycle:

Products will pass labeled AAMI Level for up to 75 processings when following above recommendations

DO NOT use fabric softener. Fabric softener inhibits absorbancy/wicking
DO NOT overdry. Overdrying can lead to melting/hardening, sheen, static and permanent creasing

*Surgical materials meet the Standard for Flammability of Clothing Textiles, CPSC 16 CFR Part 1610, Class 1 (formerly CS 191-53).



Let us know
how we
can help

Our clinical specialists can analyze your protective apparel usage trends and recommend ways to optimize utilization and purchasing to help you achieve significant savings.

Protect your team today. For more information and samples, contact your Medline Representative or visit [medline.com](https://www.medline.com).



**We make
healthcare
run better™**

Medline Industries, LP

Three Lakes Drive, Northfield, IL 60093
Medline United States | 1-800-MEDLINE (633-5463)
[medline.com](https://www.medline.com) | info@medline.com

Medline Canada

1-800-268-2848 | [medline.ca](https://www.medline.ca) | canada@medline.com

Medline México

01-800-831-0898 | [medlinemexico.com](https://www.medlinemexico.com) | mexico@medline.com

Follow us    