

# Isolation Gowns

Reduce exposure risks with all-around protection



**360°  
Protection  
by design**

# Level up for complete protection with Medline isolation gowns

When you're caring for patients and need to be at your best, the last thing you want to worry about is the performance of your protective apparel. With our AAMI Level 1 isolation gowns, feel confident you're getting the best quality, all-around protection each and every time.

- AAMI Level 1 isolation gowns
- Meets AAMI PB70 Level 1 requirements
- 360-degree protection
- Fluid-resistant and low-linting fabric




Durable




Industrial laundry  
75 wash cycles

## AAMI\* PB70:2012 requirements

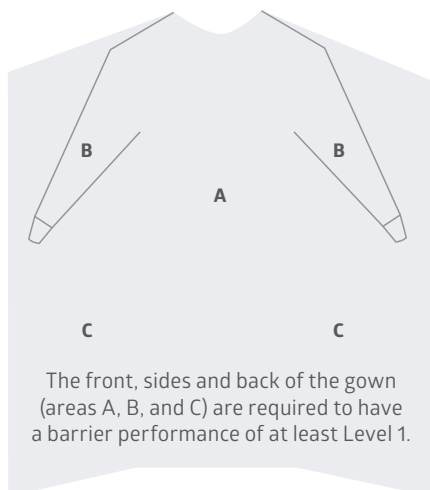

## Additional Information

|                            |                                    |
|----------------------------|------------------------------------|
| <b>Latex free:</b>         | Not made of natural rubber latex   |
| <b>Sterility:</b>          | Non-sterile                        |
| <b>Traceability:</b>       | Lot number on each case of product |
| <b>Expiration date:</b>    | No expiration date                 |
| <b>Storage conditions:</b> | General warehouse conditions       |
| <b>Fabric weight:</b>      | 2.6 OSY                            |

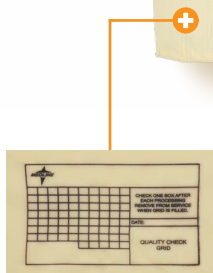

Gowns feature a quality control grid that allows you to track usage

# Level 1 fluid-resistant blockade isolation gowns

- Fluid-resistant fabric made of 100% polyester
- Overlap back
- Ties at neck and waist
- 100% polyester knit cuffs




Fluid-resistant  
100% polyester +

MDT011207



Overlap back ties at neck and waist

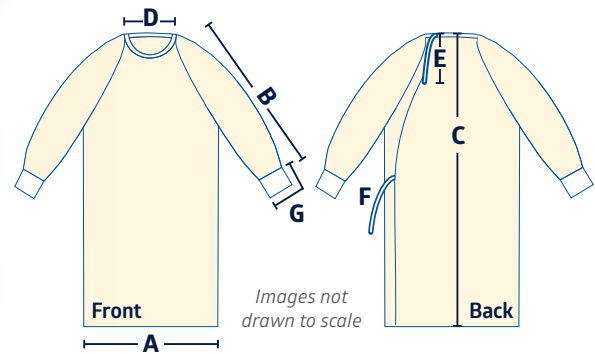

## Gown Measurements

|                        | SKU: <b>MDT011207</b> | <b>MDT011207Z</b> | <b>MDT011207XXXL</b> | <b>MDT011205</b> |
|------------------------|-----------------------|-------------------|----------------------|------------------|
| <b>Size:</b>           | Size OSFM             | Size OSFM         | Size 3XL             | Size OSFM        |
| <b>Qty:</b>            | 6 dz/cs               | 1 dz/cs           | 1 dz/cs              | 6 dz/cs          |
| <b>Gown Color:</b>     | Yellow                | Yellow            | Yellow               | Grey             |
| <b>A</b> Sweep         | 56.5"                 | 56.5"             | 69"                  | 56.5"            |
| <b>B</b> Sleeve Length | 30.5"                 | 30.5"             | 33"                  | 30.5"            |
| <b>C</b> CBL           | 44"                   | 44"               | 44"                  | 44"              |
| <b>D</b> Collar        | 20.75"                | 20.75"            | 24.25"               | 20.75"           |
| <b>E</b> Tie           | 12"                   | 12"               | 12"                  | 12"              |
| <b>F</b> Tie           | 17.5"                 | 17.5"             | 17.5"                | 17.5"            |
| <b>G</b> Cuff          | 3" x 4.5"             | 3" x 4.5"         | 3" x 4.5"            | 3" x 4.5"        |

# Level 1 fluid-resistant blockade isolation gowns with carbon fibers

- Fluid-resistant fabric made of 99% polyester/1% carbon
- Overlap back
- Ties at neck and waist
- 100% polyester knit cuffs
- Not made with PFOS/PFOA




Static-resistant




Overlap back ties at neck and waist

Fluid-resistant fabric made of 99% polyester/1% carbon

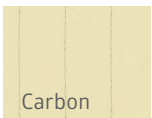

MDT011201L

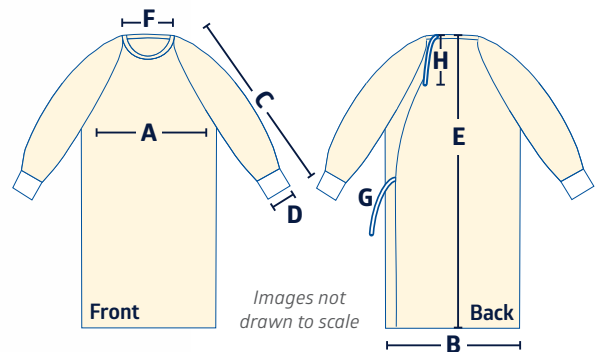


## Gown Measurements

|                         | SKU: <b>MDT011201L</b> | <b>MDT011201XL</b> |
|-------------------------|------------------------|--------------------|
|                         | Size L                 | Size XL            |
|                         | Qty: 6 dz/cs           | 6 dz/cs            |
| <b>Gown Color:</b>      | Yellow                 | Yellow             |
| <b>A</b> Chest          | 26.5"                  | 28.5"              |
| <b>B</b> Sweep          | 57.125"                | 60.375"            |
| <b>C</b> Sleeve Length  | 32.5"                  | 33.375"            |
| <b>D</b> Sleeve Opening | 6"                     | 6.125"             |
| <b>E</b> CBL            | 44.25"                 | 44.25"             |
| <b>F</b> Collar         | 21"                    | 21.5"              |
| <b>G</b> Tie            | 17.5"                  | 17.5"              |
| <b>H</b> Tie            | 12"                    | 12"                |


# Level 1 three arm hole isolation gowns

- Wraparound design makes donning and doffing easy
- Fluid-resistant fabric made of 100% polyester
- Overlap back
- No snaps or ties
- 100% polyester knit cuffs



Fluid-resistant 100% polyester 

MDTISO3AHY

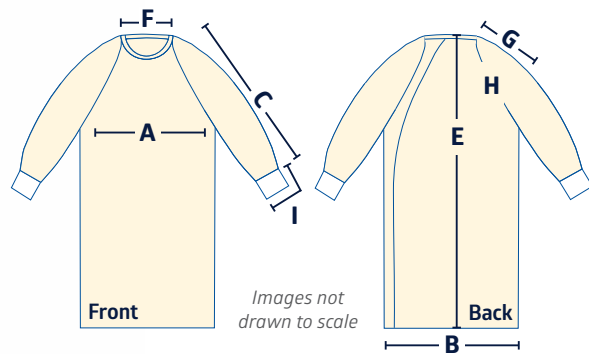

## Gown Measurements

|                         | SKU: <b>MDTISO3AHY</b> | <b>MDTISO3AHY3XL</b> | <b>MDTISO3AHG</b> | <b>MDTISO3AHG3XL</b> |
|-------------------------|------------------------|----------------------|-------------------|----------------------|
| <b>Size:</b>            | Size OSFM              | Size 3XL             | Size OSFM         | Size 3XL             |
| <b>Qty:</b>             | 6 dz/cs                | 6 dz/cs              | 6 dz/cs           | 6 dz/cs              |
| <b>Gown Color:</b>      | Yellow                 | Yellow               | Grey              | Grey                 |
| <b>A</b> Chest          | 26.5"                  | 32.5"                | 26.5"             | 32.5"                |
| <b>B</b> Sweep          | 72.75"                 | 90.5"                | 72.75"            | 90.5"                |
| <b>C</b> Sleeve Length  | 24 "                   | 24.25"               | 24 "              | 24.25"               |
| <b>D</b> Sleeve Opening | 7"                     | 8"                   | 7"                | 8"                   |
| <b>E</b> CBL            | 44"                    | 43.25"               | 44"               | 43.25"               |
| <b>F</b> Collar         | 7"                     | 7.75"                | 7"                | 7.75"                |
| <b>G</b> Shoulder       | 6.25"                  | 7.375"               | 6.25"             | 7.375"               |
| <b>H</b> Armhole        | 22.5"                  | 26.75"               | 22.5"             | 26.75"               |
| <b>I</b> Cuff           | 3" x 4.5"              | 3" x 4.5"            | 3" x 4.5"         | 3" x 4.5"            |


# Level 1 three arm hole isolation gowns with carbon fibers



Static-resistant




Fluid-resistant fabric made of 99% polyester/ 1% carbon

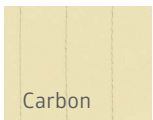

MDTISO3AHYCAR



No snaps or ties

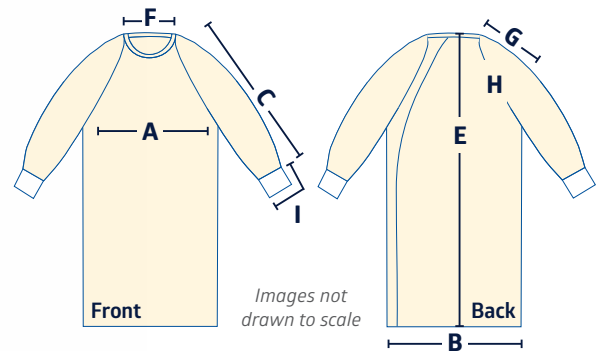

## Gown Measurements

SKU: **MDTISO3AHYCAR** **MDTISO3AHYCR3**

Size: Size OSFM Size 3XL

Qty: 6 dz/cs 6 dz/cs

Gown Color: Yellow Yellow

|                         |           |           |
|-------------------------|-----------|-----------|
| <b>A</b> Chest          | 26.5"     | 32.5"     |
| <b>B</b> Sweep          | 72.75"    | 90.5"     |
| <b>C</b> Sleeve Length  | 24 "      | 24.25"    |
| <b>D</b> Sleeve Opening | 7"        | 8"        |
| <b>E</b> CBL            | 44"       | 43.25"    |
| <b>F</b> Collar         | 7"        | 7.75"     |
| <b>G</b> Shoulder       | 6.25"     | 7.375"    |
| <b>H</b> Armhole        | 22.5"     | 26.75"    |
| <b>I</b> Cuff           | 3" x 4.5" | 3" x 4.5" |



3 ways to don and doff the Medline 3-hole isolation gown

Scan the code to [watch the video](#)

# Laundering recommendations for isolation gowns

**Formula conditions:** Washer/washer extractor, open pocket with controls for obtaining a requested temperature

**Water quality:** Soft water (zero grains hardness); low to moderate alkalinity (0–100 ppm @Na<sub>2</sub>O); no iron in water

**Wash conditions:** Steam for temperature, cold temp.: 60 °F; hot temp: 150–160 °F; loading: 90–100% of capacity

**Water levels:** Low: 0.5 gallon total water/lb.; high: 0.65 gallon total water/lb.

## Normal wash

|   | Operation | Wash temp °F | Wash time | Water level | Approx. pH | Quantity fl. oz/cwt         |
|---|-----------|--------------|-----------|-------------|------------|-----------------------------|
| 1 | Flush     | 150 °F       | 5 min.    | Low         |            |                             |
| 2 | Break     | 150 °F       | 9 min.    | Low         | 10.8–11.2  | 6–8 (Alkali), 4 (Detergent) |
| 3 | Rinse     | 150 °F       | 10 min.   | Low         |            |                             |
| 4 | Rinse     | 150 °F       | 7 min.    | High        |            |                             |
| 5 | Rinse     | 110 °F       | 7 min.    | High        |            |                             |
| 6 | Sour      | 90 °F        | 5 min.    | Low         | 5.5–6.5    | 1–2 (Sour)                  |
| 7 | Extract   |              | 3 min.    |             |            |                             |

**Finished fabric condition:** pH 5.5–6.5

## Bleach

**Do not bleach**—may cause damage to barrier properties

## Normal

**Tumble dry:** Max 160 °F

**Do not over dry.** Can lead to melting, hardening, sheen, static, and permanent creasing

## Iron

**Do not flat-work iron**

### All products:

Wash prior to use

Fabric softeners not recommended; softeners may inhibit the barrier properties

A thorough rinse is essential to remove all surfactants; residual surfactants may adversely affect the barrier properties

### Chemicals:

Quantity will vary with machine size and type

Confirm with chemical supplier for correct quantity



Let us know  
how we  
can help

Our clinical specialists can analyze your protective apparel usage trends and recommend ways to optimize utilization and purchasing to help you achieve significant savings.

**Protect your team today.** For more information and samples, contact your Medline Representative or visit [medline.com](https://www.medline.com).



**We make  
healthcare  
run better™**

**Medline Industries, LP**  
Three Lakes Drive, Northfield, IL 60093  
Medline United States | 1-800-MEDLINE (633-5463)  
[medline.com](https://www.medline.com) | [info@medline.com](mailto:info@medline.com)

**Medline Canada**  
1-800-268-2848 | [medline.ca](https://www.medline.ca) | [canada@medline.com](mailto:canada@medline.com)  
**Medline México**  
01-800-831-0898 | [medlinemexico.com](https://www.medlinemexico.com) | [mexico@medline.com](mailto:mexico@medline.com)

Follow us    