


Single-Head Aluminum Stethoscopes



Maintain superior acoustic sound with proper use and care.

Instructions for use

Adjust the headset

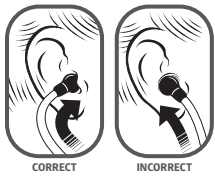


Easily find your most comfortable fit.

- Provides a comfortable, acoustically sealed ear fit
- Ear tubes are shaped to fit right in the ear canal

- Ear tips should point in a forward direction as you insert them into the ear canal


For optimal acoustic performance

- Use soft-sealing ear tips provided with your stethoscope
- Ear tip attaches to the ear tube through a unique design
- Ear tips are pushed onto the end of the ear tube and snapped firmly into place
- To remove, pull firmly on the ear tip

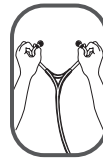

To reduce spring tension

- Hold each ear tube at the bend near the ear tip and gradually pull apart until fully extended

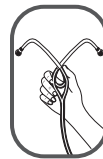

To increase spring tension

- Grasp the headset with one hand where the ear tubes enter the tubing
- Squeeze until the tubing on one ear tube touches the other. Repeat as necessary

Everyday care

- Keep stethoscope out of extreme heat and cold
- Clean with a non-bleach alcohol wipe or wipe down with soapy water
- Do not immerse stethoscope in any liquid

Learn more. Contact your Medline Representative or call 1-800-MEDLINE for more information.