

Donning and Doffing


Hook and loop & tape tab disposable gowns.



Donning


- Shake out the gown to easily access the arm and head openings.
- Place your arms in the sleeves and hands through the wrist openings.
- Secure the gown by closing the gown at the neck with the hook and loop or tape tab, making sure it's securely fastened.
- Tie the gown at the back at the waist or hip to ensure the gown is properly secured.

Note: If donning a full-back gown, ensure your back is completely covered.

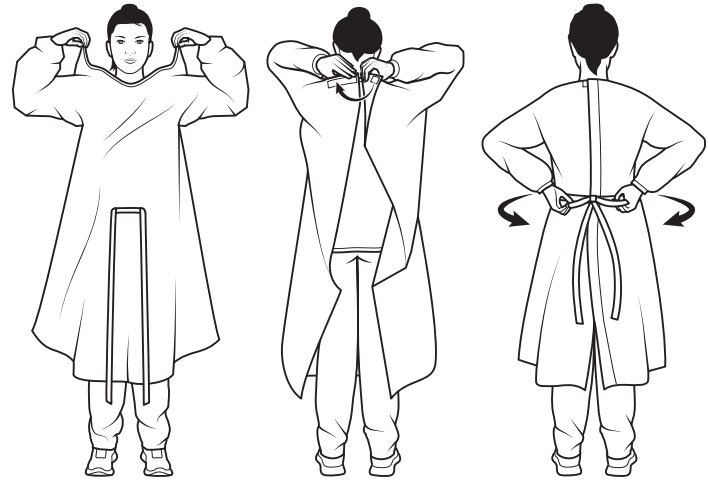

Doffing

- To remove the gown, grip it at each hip and pull up and out, breaking the gown away from your back and neck.
- Continue rolling the gown down and away from the body, avoiding contact with the soiled gown.
- As you reach the wrists, simultaneously remove your gloves and roll them into the gown as one entity.
- Discard in the trash or biohazard bin as appropriate.

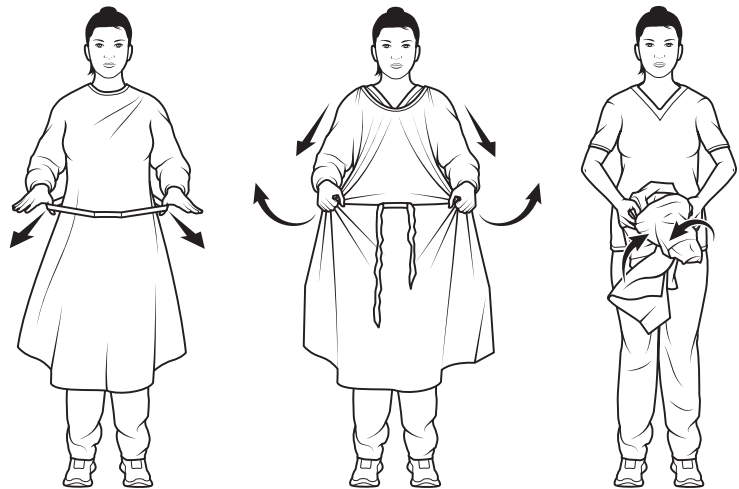

Scan here to view instructional video.

Medline Industries, LP | 1-800-MEDLINE (633-5463) | www.medline.com

© 2025 Medline Industries, LP. All rights reserved. Medline is a registered trademark of Medline Industries, LP. All product names, regardless of whether or not they appear with a trademark symbol, are the property of their respective companies unless otherwise noted.
MKT19W8062130 / e25336 / 55

Learn more

Contact your Medline Representative or call **1-800-MEDLINE**