

Donning and Doffing


Neck tie disposable gowns



Donning


- Shake out the gown to easily access the arm and head openings.
- Place your arms in the sleeves and hands through the wrist openings.
- Secure the gown by tying the neck in a tight bow to ensure the gown is properly secured.
- Tie the gown at the back at the waist or hip to ensure the gown is properly secured.

Note: If donning a full-back gown, ensure your back is completely covered.

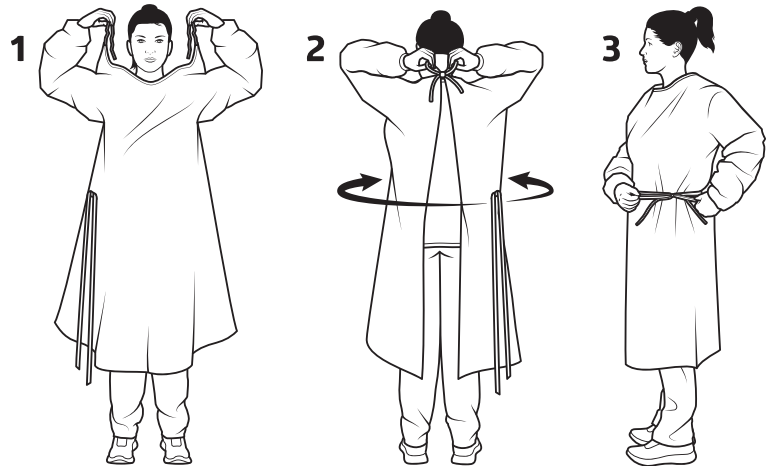

Doffing

- To remove the gown, carefully untie the neck tie and then pop the waist tie off the gown. Once the waist tie has been released, grip it at each hip and pull up and out, breaking the gown away from your back.
- Continue rolling the gown down and away from the body, avoiding contact with the soiled gown.
- As you reach the wrists, simultaneously remove your gloves and roll them into the gown as one entity.
- Discard in the trash or biohazard bin as appropriate.

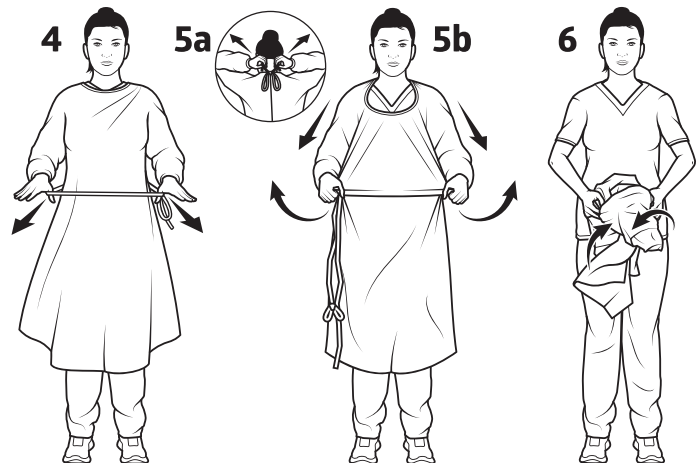

Scan here to view instructional video.

Learn more

Contact your Medline Representative or call **1-800-MEDLINE**