Folding Commode Chaise d'aisance pliante

Silla retrete plegable

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General Warnings

To ensure the safe assembly and operation of this product, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- **DO NOT** assemble and use this product without first reading and understanding this user's guide, otherwise injury or damage may occur.
- **DO NOT** use if components are missing or are damaged. Contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.
- **DO NOT** exceed weight capacity of **350 lb (158 kg)**.
- **DO NOT** make any adjustments while using the product.
- **DO NOT** stand on the product.
- **DO NOT** attempt to use the arms to support full body weight.

Cautions:

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following:

1. Toilet seat must be in the down position before use. Failure to do so may result in possible injury or damage.
2. All push buttons are fully protruded and are properly engaged.
3. There are no loose, broken or missing components.
4. All parts are secure.
5. When using the backrest, ensure the commode remains balanced. Excessive twisting or turning while seated on the commode may cause a loss of balance and/or damage to the product.
6. All four legs touch the floor simultaneously at all times.
7. User evenly distributes weight between the arm supports when transferring in and out of the commode.
8. Armrests are dry.
9. The tips are not worn, broken or missing. Replace any missing or damaged tips before use.

When applicable:

10. The wheels and wheel locks are not worn, broken or missing. Replace any missing or damaged parts before use.
11. The wheel locks are properly engaged during use.
12. Arms are unlatched and dropped to assist in transfer on or off the commode.

Assembly:

If exposed to cold temperatures, allow contents to come to room temperature before completing assembly. Remove contents from box, ensuring all parts listed are included:

1. Frame
2. Bucket w/ Lid
3. Splashguard
4. Seat and Lid

1. Before assembling seat and lid, allow toilet seat and lid to reach room temperature if exposed to cold. This will help prevent the seat and lid clamps from cracking while being assembled.
2. Align the toilet seat clamps onto the back frame rail. (Both clamps should sit outside the pail brackets.) Push down on seat until clamps snap firmly into place over frame rail.
3. Repeat the same process for the lid. (Note: G1-202EX4 and G1-202ELX4 do not come with lid.)
4. When removing the seat or lid, always remove lid first and then the seat. To remove, grasp the back of the lid and/or seat and lift.
5. Install commode bucket by dropping into bucket frame. Lift bucket from frame to remove for emptying and cleaning.

Leg Height Adjustment

1. Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height. When your desired height is reached, lock legs into place by aligning buttons with holes until they “snap” into place (see **Figure 1**).
2. Make sure legs are locked into place by pushing down or by pulling up on legs.

Figure 1

Maintenance

Periodic visual inspection of this product is recommended to ensure that all parts and hardware are secure, that components are in good working order and not cracked, broken or fractured, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.

Cleaning

1. Clean with regular household products. Use a non-abrasive cleaner with warm water. Wipe dry after cleaning.
2. **DO NOT** use abrasive powders, cleaners or scouring pads.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as tip accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual.

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