

McKesson

MULTI-ENZYMATIC FOAMING CLEANSER – RTU

- **An effective pre-decontamination spray for surgical instruments and scopes**
- **Effectively breaks down blood, proteins, fat, carbohydrates and other particles**
- **Neutral pH, non-abrasive, and 100% biodegradable**
- **Safe for use on all surgical instruments and scopes**

McKesson Multi-Enzymatic Foaming Cleanser-RTU is a ready-to-use spray for pre-cleaning surgical instruments and scopes. Its thick foaming spray effectively covers the instruments and scopes, cleaning away blood, tissue, mucous and other protein-rich body fluids. **Multi-Enzymatic Foaming Cleanser-RTU** is easy to use, and saves time by starting the pre-cleaning process right away.

DIRECTIONS FOR USE:

1. Spray product onto instruments and scopes immediately after use.
2. Hold sprayer 6 to 8 inches from instruments and scopes and spray thoroughly, ensuring that they are completely covered with product. Apply more as needed to keep instruments and scopes moist.
3. Allow instruments and scopes to soak in foam until ready for cleaning.
4. Rinse instruments and scopes before proceeding with the decontamination process.

INGREDIENTS:

Purified Water, Propylene Glycol, Anionic Surfactants, Nonionic Surfactants, Proteolytic Enzymes, Fragrance, Citric Acid and Blue 1

PACKAGE SIZES:

32 Ounce Bottle (12/case)

Reorder No.

53-28517

PRODUCT CHARACTERISTICS:

PRODUCT DESCRIPTION:	Water based ready-to-use foaming enzymatic cleaner
CLARITY:	Clear
COLOR:	Blue
ODOR:	Pleasant
SPECIFIC GRAVITY:	1.00 to 1.04
DENSITY:	8.34 to 8.67 pounds per gallon
QUALITY:	Manufactured in accordance with FDA current Good Manufacturing Practices (cGMP's)

McKesson Corporation

McKesson Medical-Surgical
8741 Landmark Rd.
Richmond, VA 23228