

Mechanical Floor Scale



Instructions for Use



REF MDR300FM | Not made with Natural Rubber Latex

How to Use

1. Place the scale on a hard, flat surface to ensure accuracy. Make sure there are no objects beneath the scale before weighing.
2. Gently step on the scale a few times before weighing yourself, this will set the internal mechanism.
3. If the scale pointer is not at zero when no weight is applied (Image A), move the adjustment knob (Image B) gently to the right, if you do not see "0", turn the adjustment knob gently to the left.
4. Step on the scale with weight equally distributed on both feet. Stand still to take a measurement.

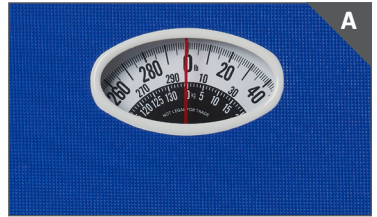


Image A



Image B

Maintaining the Scale:

1. The baked enamel finish, lens and mat can be cleaned with a mild soap and damp cloth. **DO NOT USE ABRASIVE OR STRONG CHEMICAL CLEANERS.**
2. All bearings and working parts are permanently lubricated. **DO NOT USE OIL OR LUBRICANTS.**
3. If your scale fails to work properly, do not attempt to dismantle it. This will void your warranty.

Warranty:

One year limited warranty against manufacturers defects.



Medline Industries, LP, Three Lakes Drive, Northfield, IL 60093 USA
Medline United States
1-800-MEDLINE (633-5463)
medline.com | info@medline.com

Medline Canada
1-800-268-2848 | medline.ca | canada@medline.com

Medline Mexico
01-800-831-0898 | medlinemexico.com | mexico@medline.com

Follow us

