



MEDIGRIP™ & MEDIGRIP™ LF

Application guide

Elastic tubular bandage

MEDIGRIP™ provides support for joints, or mild compression when applied as a double layer and can help secure dressings.




Scan code to see application video or visit www.medline.com/awcvideos

Application:



1. Measure the area with the largest circumference. Use the sizing chart to determine the appropriate sized bandage to deliver the desired compression and cut to the desired length.

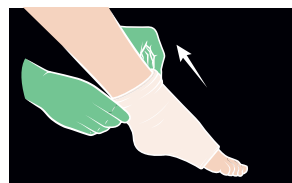

2. Draw the bandage gently and evenly over the limb into place, covering the area like a sleeve.



3. If greater compression is required, cut double the needed length. First, draw one end of the bandage over the limb to cover the area. Then, pull the other end of the bandage over the first to create a folded double layer.

CONTRAINDICATIONS: Exercise caution when applying compression to patients with severe arterial disease. Always conduct a patient assessment to ensure the safety of compression therapy.

See product packaging for more information.

© 2021 Medline Industries, Inc. Medline is a registered trademark and Medigrip is a trademark of Medline Industries, Inc. MKT19W1633764 / e20020 / 55

Elastic tubular bandage