



**Medline Industries, Inc.**  
3 Three Lakes Dr.  
Northfield, IL 60093

1.847.949.5500  
1.800.MEDLINE (633.5463)

[medline.com](http://medline.com)

## **Cleaning Instructions for Medline's Reusable Fluid Proof Pillows**

The following instructions pertain to: Nylex, Medsoft, Ovation, Stay-Fluff, and DWR pillows.

Pillows should be wiped down with disinfectants and fully dry before use.

Disinfectants should be diluted as specified on label of manufacturer. As recommended by the Center for Disease Control and Prevention, a 1:10 bleach dilution may be used.

*Laundering is not recommended and will damage the pillows.*