



OPTIFOAM[®] GENTLE AG⁺ NON-BORDERED

SILICONE FACED FOAM

ANTIMICROBIAL SILICONE FACED FOAM




PREPARATION:


Step 1: Cleanse wound environment using an appropriate wound cleanser.

Step 2: Ensure skin surrounding wound is dry and apply appropriate skin prep, as preferred.

APPLICATION:

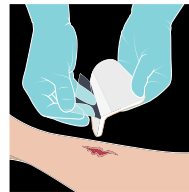

1. Remove dressing from package. Remove one piece of the release liner. Note: dressing may be cut to fit.

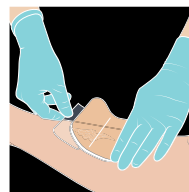

2. Place dressing on the wound and remove the other release liner.



3. Secure in place using tape or a secondary dressing such as elastic net.

See application video at www.medline.com/awcvideos

© 2017 Medline Industries, Inc. Medline and Optifoam are registered trademarks of Medline Industries, Inc. MKT1790249 / e17272

SILICONE FACED FOAM DRESSING