

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

oxygen cylinder stand
for h/m cylinders



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

instructions

1. Place cylinder stand on the floor.
2. Insert oxygen cylinder, making sure base is resting on the floor and cylinder is in an upright position.
3. To secure oxygen cylinder, tighten screws by turning clockwise. Make sure the screws are tightened the same amount to ensure that the cylinder is balanced.
4. To remove, turn screws counterclockwise.



drive
MEDICAL DESIGN & MANUFACTURING

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

drive
MEDICAL DESIGN & MANUFACTURING