




Skintegrity®

Surfactant-Based Wound Cleanser


Skintegrity® Wound Cleanser provides gentle and thorough dermal wound cleansing.




Application:

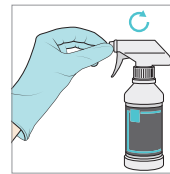

1. Turn nozzle to spray or stream position.

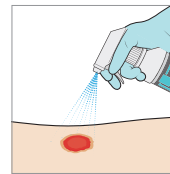

2. Spray cleanser on wound from approximately 3 inches away. Trigger spray provides an ideal PSI per AHRQ guidelines.

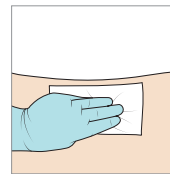

3. Wipe away any debris and pat wound dry. Return nozzle to off position. Apply as needed.

See application video at www.medline.com/awcvideos

See product packaging for more information.
© 2020 Medline Industries, Inc. Medline and Skintegrity are registered trademarks of Medline Industries, Inc.
MKT19W628509 / e20202 / 3

Wound Cleanser (8 and 16 oz)