



Cleaning instructions for Medline's reusable fluid proof pillows.

The following instructions pertain to: Nylex, Medsoft, Ovation, and Stay-Fluff pillows

Routine Care:

Wipe clean with neutral soap suds and lukewarm water. Rinse and dry.

Hard to Clean Areas:

Use standard vinyl cleaner and soft sponge.

Solvents, harsh cleaners and detergents should not be used.

Disinfectants:

Germicides and disinfectants should be diluted as specified on label of manufacturer. As recommended by the Centers for Disease Control, a dilution of household bleach may be used.

Laundering is not recommended and will damage the pillows.