


Drugs of Abuse Test Cups

MULTI-DRUG PANEL WITH ADULTERANTS


CLIA WAIVED

For use with MFR #s 16-5125A3, 16-6125A3, 16-1145A3 and 16-8105A3.

Consult instructions for use 

For *in vitro* diagnostic use only **IVD**

General Questions? Call 1-800-777-4908
Technical Support? Call 1-866-982-3818

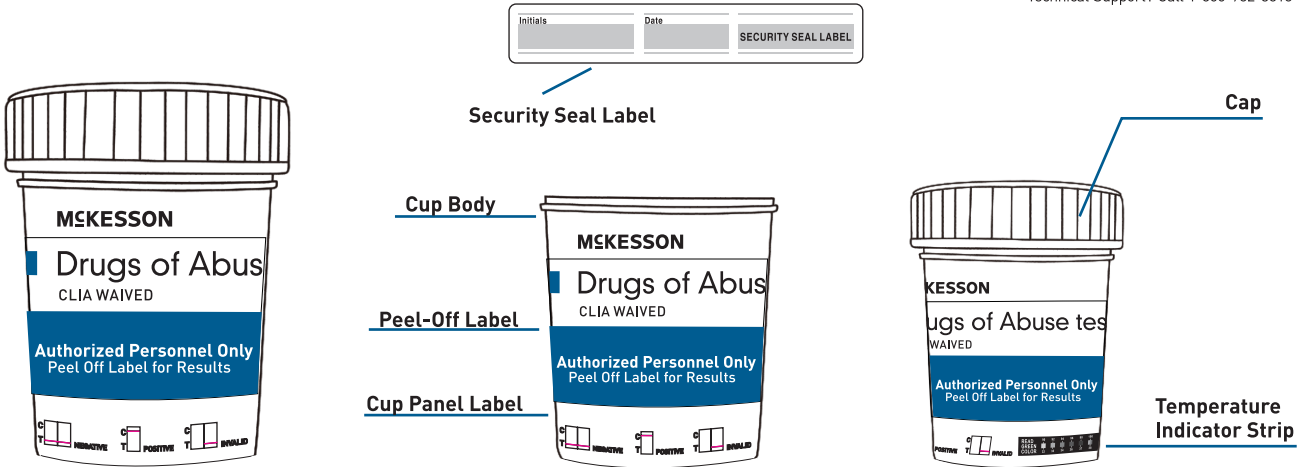

- 1 Donor provides urine specimen in cup. If desired, cap can be held by operator to initiate the test at a later time.
- 2 Once urine sample has been collected, screw cap firmly onto cup.
- 3 Check urine temperature at 2-4 minutes by reading green color on temperature indicator strip.



- 4 Donor dates and initials security seal. Operator checks cap for tightness and attaches security seal label over cap as shown (optional).
- 5 Remove peel-off label.
- 6 Read adulteration strip pads between 2 minutes, if applicable.



- 7 Read test results at 5 minutes. Refer to diagram below for result interpretation.
- 8 Positive test results must be confirmed by another test method. Send cup and urine specimen with security seal label intact to a toxicology laboratory for confirmation.

