

+ REPLICARE[◊] M and REPLICARE[◊] M THIN Hydrocolloid Dressing

Soft hydrocolloid dressing, designed to maintain a moist wound environment, for the treatment of non-exuding to moderately exuding wounds.*








Smith+Nephew

REPLICARE[◊] M
Hydrocolloid Wound Dressing

REPLICARE[◊] M THIN
Thin Hydrocolloid Wound Dressing

www.smith-nephew.com

EXUDATE LEVEL	PRIMARY INDICATIONS	OTHER INDICATIONS
 NON TO MODERATE ABSORBENCY	 BURNS (SUPERFICIAL AND SUPERFICIAL PARTIAL THICKNES)  POST-OPERATIVE WOUNDS	SKIN ABRASIONS MINOR CUTS LACERATIONS
 NON TO LOW ABSORBENCY	 ACUTE WOUNDS	SCALDS

REPLICARE[◇] M and REPLICARE[◇] M THIN Designed to maintain a moist wound environment

Features and benefits



Excellent absorption

REPLICARE M and M THIN Dressings' fluid handling capacity allows the dressing to remain in place for up to 7 days.¹



Soft

The dressing's material interacts with the wound exudate, forming a soft gel.



Waterproof

REPLICARE M and M THIN Dressings are waterproof, helping patients to continue with day-to-day activities.¹




Protective barrier

REPLICARE M and M THIN Dressings help protect the wound from germs and dirt.^{1,2}



Visibility of wound progress

The dressing's semi-transparent top film allows the wound's healing progression to be monitored, without the need for unnecessary dressing removal.[†]

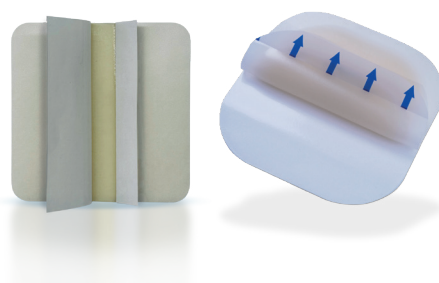

Easy application in 4 simple steps:

1. Cleanse the application site.
2. Select an appropriate size dressing to cover the wound area by at least 2-3 cm. If necessary, cut the dressing to shape.
3. Apply the adherent side of the dressing to the wound, and smooth the dressing in place.
4. To remove the dressing, gently lift one edge. Gently loosen the edges until completely removed from the wound..



Ordering information

REPLICARE[◇] M Hydrocolloid Wound Dressing

Order No.	Description	Each/Box	Box/Case
66807707	3in x 3in	10	40
66807708	4in x 4in	10	40
66807709	6in x 6in	5	20

Ordering information

REPLICARE[◇] M THIN Thin Hydrocolloid Wound Dressing

Order No.	Description	Each/Box	Box/Case
66807710	2in x 6in	10	40
66807711	6in x 8in	10	15
66807712	4in x 4in	10	40
66807713	6in x 6in	5	30

For detailed product information, including indications for use, contraindications, warnings and precautions, please consult the product's Instructions for Use (IFU) prior to use.

*REPLICARE M THIN Dressing is indicated for non-exuding to low exuding wounds. REPLICARE M Dressing is indicated for non-exuding to moderately exuding wounds.
[†]Only applicable to REPLICARE M THIN Dressing.

References: 1. Replicare M: SYSQ20240620001-03. 2. Replicare M Thin: SYSQ20240620001-02..

Advanced Wound Management, Smith & Nephew, Inc. Fort Worth, TX 76109 USA ©2024 Smith+Nephew. ◇Trademark of Smith+Nephew. All Trademarks acknowledged. | SCPE8-42542-0924