

Puracyn® Plus Wound and Skin Care Solution, OTC Formula

McKesson Puracyn® Plus wound and skin care solution streamlines the process of caring for minor wounds and sores at home. The over-the-counter formula uses the Puracyn Plus technology at a strength appropriate to effectively clean minor cuts, scrapes, sores and abrasions and prepare them to heal.

Puracyn Plus Technology

Puracyn Plus technology uses an advanced, super-oxygenated, pH-balanced hypochlorous solution to help set the stage for optimal healing conditions.

Indications for Use

The wound and skin care solution is intended for OTC use in the management of minor skin wounds including minor lacerations, abrasions, irritations, cuts, burns, and intact skin.

Features and Benefits

- Non-toxic and non-irritating
- Contains no antibiotics or steroids
- Contains no alcohol or iodine
- Won't harm healthy tissue
- Environmentally friendly, no special disposal required
- Not made with the natural rubber latex



Satisfaction Guaranteed

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit.

Never Compromise on CareSM

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them—making it even easier for you to provide the best care for your patients.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.

Puracyn and Puracyn Plus are registered trademarks of Innovacy, Inc.

© 2019 McKesson Medical-Surgical Inc.

MICROBIAL INHIBITION

In vitro testing of the preservative in the Puracyn Plus technology shows microbial inhibition activity for a broad range of microorganisms including staphylococcal species.

ORGANISMS	MICROBIAL REDUCTION (15 SECONDS)
<i>K. pneumoniae</i>	99.99999%
<i>E. faecalis</i>	99.99999%
<i>S. aureus MRSA</i>	99.99999%
<i>S. epidermidis</i>	99.99999%
<i>A. baumannii</i>	99.99999%
<i>S. aureus</i>	99.99999%
<i>P. aeruginosa</i>	99.99999%
<i>E. coli</i>	99.99999%
<i>C. albicans</i>	99.99999%

Source: Innovacyn Technical Report ITR017-1

COMPARATIVE CYTOTOXIC ANALYSIS

Some wound cleansing formulations have been shown, *in vitro*, to impair the functioning of the cells involved in normal wound healing.¹ On the other hand, the use of saline or tap water is inappropriate for wound bed preparation in the context of bioburden and biofilm.²

WOUND IRRIGANT	RESULTS
Saline (0.9%, NaCl, pH 5.0)	Pass
Hypochlorous Acid (0.012%)	Pass
Sodium Hypochlorite (Dakin's) (0.25%)	Fail
Sodium Hypochlorite (Dakin's) (0.5%)	Fail
Chlorhexidine Gluconate (4%)	Fail
Hydrogen Peroxide (3%)	Fail
Povidone Iodine (7.5%)	Fail

1 Wolcott R, Fletcher J. The role of wound cleansing in the management of wounds. *Wounds International*. 2014; 1(1): 25-31.

2 Schultz GS, Sibbald RG, Flanagan V, Ayello EA. Wound bed preparation: a systematic approach to wound management. *Wound Rep Reg*. 2003; 11: 1-28.

ORDERING INFORMATION

MFR #	DESCRIPTION	ATTRIBUTES	APPLICATOR	NET CONTENTS	HCPCS	QTY
186-6002	Wound and Skin Care Solution	Non-Sensitizing, Non-Toxic, Non-Irritating, Single Patient Use, Over-the-Counter	Pump Spray	8.5 fl oz	N/A	1 ea, 6 ea/cs
186-6004	Wound and Skin Care Solution	Non-Sensitizing, Non-Toxic, Non-Irritating, Single Patient Use, Over-the-Counter	Trigger	16.9 fl oz	N/A	1 ea, 6 ea/cs

McKesson Medical-Surgical
9954 Mayland Drive
Richmond, VA 23233
mms.mckesson.com/mckesson-brands

Post-Acute Care and Home Health
800.347.2456

Home Medical Equipment
888.822.8111

Primary Care
866.625.2679

McKesson Brands Support
800.777.4908

