

Collagen dressings with silver

The McKesson collagen dressings with silver are advanced wound care dressings made of collagen, sodium alginate, carboxymethylcellulose (CMC), ethylenediaminetetraacetic acid (EDTA), and silver chloride. The topically applied wound dressing will transform into a soft gel sheet when in contact with wound exudates. The dressing maintains a moist wound environment at the wound surface that aids in the formation of granulation tissue and epithelialization. The EDTA in the dressing is intended to help maintain dressing integrity. The silver chloride in the dressing is an effective antimicrobial intended to prevent colonization of the dressing. The dressings may be trimmed and layered for the management of deep wounds.

INDICATIONS FOR USE: The McKesson collagen dressings with silver is indicated for management of full and partial thickness wounds including pressure ulcers, diabetic ulcers, ulcers caused by mixed vascular aetiologies, venous ulcers, donor and graft sites, abrasions, traumatic wounds healing by secondary intention, dehisced surgical wounds, and first and secondary degree burns.

CONTRAINDICATIONS: The McKesson collagen dressings with silver should not be used on third degree burns or on patients with a known allergy or sensitivity to fish-derived collagen, silver, or any other ingredient.

PRECAUTIONS: If sensitivity to the dressing develops, discontinue use. Dressings may be used with compression therapy under the supervision of a health care professional.

ACTIVE INGREDIENT: Silver chloride to reduce the incidence of bacterial colonization within the wound dressing.

Features and Benefits

- Silver chloride helps to reduce the incidence of bacterial colonization within the wound dressing
- Facilitates natural wound healing
- Helps maintain a moist wound environment
- Dressing can be cut to fit the wound size
- Dressing change frequency - daily (or possibly up to 7 days depending on wound type)
- Not made with natural rubber latex



Satisfaction Guaranteed

For complete details, please visit mms.mckesson.com/mckesson-brands.

Never Compromise on Care™

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them—making it even easier for you to provide the best care for your patients.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.

© 2025 McKesson Medical-Surgical Inc.

DIRECTIONS FOR USE

1. Debride and irrigate the wound bed with an appropriate wound cleanser or distilled water before application. The use of saline may affect the efficacy of the silver.
2. Cut the dressing to fit the exact wound size. For heavily exudating wounds, apply the dressing directly to the wound bed. For dry wounds with very little exudate, moisten the wound bed with distilled water to begin the gelling process.
3. Cover the McKesson collagen dressing with silver with an appropriate secondary dressing.
4. Change dressings daily or as good nursing practice dictates.

REMOVAL

1. Remove the secondary dressing with care. Gently remove McKesson collagen dressings with silver dressing ensuring that the dressing removal does not damage any newly formed tissues.
2. Use an appropriate wound cleanser or distilled water to cleanse the wound site prior to applying a new McKesson collagen dressings with silver as per application instructions.



ORDERING INFORMATION

IMAGE	E1 #	MFR #	DESCRIPTION	HCPCS	QTY
	1172341	16-1860	Collagen Dressings with Silver, 2 in x 2 in	A6021	10 ea/bx, 10 bx/cs
	1172342	16-1862	Collagen Dressings with Silver, 4 in x 4 in	A6021	10 ea/bx, 10 bx/cs