

2 Layer Compression Bandage System

The McKesson 2 layer compression bandage system with a diamond indicator consists of a white, padded bandage and a brown, cohesive compression bandage featuring a diamond print. This is a non-sterile product for single use only. The 2 layer compression bandage system is indicated for long-term compression therapy for venous diseases and edema caused by other factors (in patients with an ABPI* of 0.8–1.3). Positioning the two bandages in a layered combination forms a compression system, which can be worn for up to a week, depending on the condition of the wound and the time for which the wound dressing is worn. The product must be used on intact (uninjured) skin only. Wounds must be treated with a suitable wound dressing in advance. Due to its short-stretch properties, the 2 layer compression bandage system has a high working pressure and a low resting pressure (in mobile patients). As a result, edema can be reduced. The 2 layer compression bandage system can be used by medical specialists in both clinical and domestic settings.

Layer 1: White - Approx. 85% polyester 8% elastane 7% synthetic rubber

Layer 2: Brown - Approx. 62% polypropylene 22% elastane 16% synthetic rubber

LAYER 1: COMFORT PADDING BANDAGE

Redistributes pressure to aid in the management of venous insufficiencies.

4 in x 7.1 yds (10 cm x 6.5 m)

STRETCHED

LAYER 2: COHESIVE COMPRESSION BANDAGE

Provides compression (30-40mmHg) and helps keep the bandage in place for up to 7 days.

4 in x 6.9 yds (10 cm x 6.3 m)

STRETCHED



VISUAL DIAMOND INDICATOR

Shows when standard compression of 30-40mmHg has been achieved





2 Layer Compression Bandage System

Features and Benefits

- Multilayer compression system to be used on intact skin
- Wounds must be treated with a suitable wound dressing in advance
- Bandage system simultaneously absorbs, protects, and provides sustained therapeutic compression for up to seven days when applied as directed
- Visual diamond indicator on cohesive bandage helps with correct application
- Diamond indicator forms a square signaling sufficient stretch to achieve the required compression pressure
- Short stretch properties yield high working pressure and low resting pressure (in mobile patients); helps reduce edema
- Breathable
- Both layers are tearable by hand
- Standard compression rating 30-40 mmHg
- Wearing interval up to 7 days
- Both bandages must be used together to provide a secure, thin, non-slip compression dressing
- Not made with natural rubber latex



E1 #	MFR #	COMPONENTS	DESCRIPTION	QTY
1256549	2006	Compression Bandage System with Diamond Indicator Multilayer compression system (30-40 mmHg) to be used on intact skin	2 LAYER NON-STERILE	1/KIT 8 KITS/CASE
		Layer 1: White comfort padding bandage Approx. 85% Polyester 8% elastane 7% synthetic rubber Redistributes pressure to aid in the management of venous insufficiencies	4 in x 7.1 yds (10 cm x 6.5 m) Stretched	1 PER PACK
		Layer 2: Cohesive compression bandage Approx. 62% Polypropylene 22% Elastane 16% Synthetic Rubber Provides compression (30-40mmHg) and helps keep the bandage in place for up to 7 days	4 in x 6.9 yds (10 cm x 6.3 m) Stretched	1 PER PACK

Satisfaction Guaranteed
For complete details, please visit mms.mckesson.com/mckesson-brands.

Never Compromise on Care™

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them—making it even easier for you to provide the best care for your patients.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.