

## IFOBT TEST

*For in vitro diagnostic use*

McKesson Consult® iFOBT Test is a rapid immunochemical fecal occult blood test designed to detect the presence of blood in stool specimens. It uses antibodies to detect human hemoglobin protein and can be a more accurate testing tool than a standard FOBT test. Due to its high specificity for hidden blood, the McKesson Consult iFOBT Test helps to ensure effective screening for colorectal cancer. High sensitivity allows for early detection of gastrointestinal bleeding disorders, particularly those in the colon and rectum. This test can be used for in vitro diagnostics only.

McKesson Consult iFOBT Tests can be taken home at the convenience of the patient. Instructions are included to help simplify understanding of the test procedure. USPS compliant packaging is designed to provide safe and efficient specimen transport. Mailing return envelopes include: 1 sample collection slide, 2 applicator sticks, 2 collection tissues, patient instructions and package insert.

## FEATURES AND BENEFITS

- 5-minute result time
- CLIA waived
- Specific to human hemoglobin
- Easy-to-use for patient and caregiver alike
- Take home test option for patient convenience
- Not made with natural rubber latex



### Satisfaction Guaranteed

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit.

### Never Compromise on Care<sup>SM</sup>

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them—making it even easier for you to provide the best care for your patients.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.

© 2017 McKesson Medical-Surgical Inc.

**Directions for Use**

DIAGNOSTICS COLLECTION SLIDE : PATIENT INSTRUCTIONS




1. Lift toilet seat and position provided sample collection paper across rim of toilet bowl.



2. Deposit bowel movement onto collection paper. Collect sample from 4 different sites of the stool using one of the provided applicator sticks.



3. Place sample card on a flat surface, open flap of Consult sample card. Apply sample to window 1. Spread evenly and fill window with stool sample. Close flap, flush tissue.

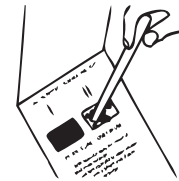


4. Repeat for steps 2 through 4 for second bowel movement and apply to collection window 2. Close the flap.




5. Insert completed Consult sample card into mailing pouch and return to your healthcare provider.


DIAGNOSTICS COLLECTION SLIDE : PHYSICIAN INSTRUCTIONS

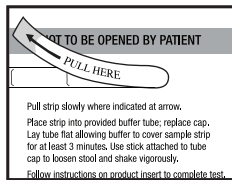

1. Slowly peel back the sample tab.

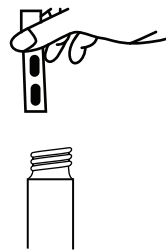

2. Place sample strip into preparation buffer tube, replace cap and shake tube vigorously.

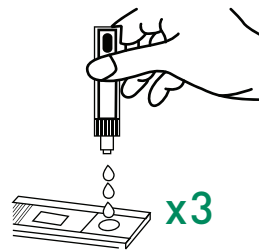


3. Squeezing the tube slightly, dispense three (3) drops into cassette well.



4. Read result at five (5) minutes.

RESULT READING:

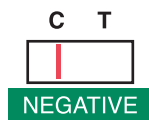

DESCRIPTION

ADDITIONAL INFO

MFR#

QTY

DESCRIPTION	ADDITIONAL INFO	MFR#	QTY
iFOBT Occult Blood Testing	25 cassettes, 25 preparation buffer tubes, 35 mailing return envelopes*	4485	25ea/bx
iFOBT Cassette Kit	10 cassettes, 10 preparation buffer tubes	4486	10ea/bx
iFOBT Cassette Kit	25 cassettes, 25 preparation buffer tubes	4487	25ea/bx
iFOBT Mailer Envelopes	USPS compliant packaging for safe and easy shipping	4488	20ea/bx

\*Mailing return envelopes include: 1 sample collection slide, 2 applicator sticks, 2 collection tissues, patient instructions, procedure card, and package insert.