

Collagen dressings

The McKesson collagen dressing is an advanced wound care dressing made of collagen, sodium alginate, carboxymethylcellulose (CMC), and ethylenediaminetetraacetic acid (EDTA). The topically applied wound dressing will transform into a soft gel sheet when in contact with wound exudates. The dressing maintains a moist wound environment at the wound surface that aids in forming granulation tissue and epithelialization. The EDTA in the dressing is intended to help maintain dressing integrity. The dressings may be trimmed and layered to manage deep wounds.

INDICATIONS FOR USE: The McKesson collagen dressings are indicated for the management of full-and partial-thickness wounds, including pressure ulcers, diabetic ulcers, ulcers caused by mixed vascular aetiologies, venous ulcers, donor and graft sites, abrasions, traumatic wounds healing by secondary intention, dehisced surgical wounds, and first-and second-degree burns.

CONTRAINDICATIONS: The McKesson collagen dressing should not be used on third-degree burns or on patients with a known allergy or sensitivity to fish-derived collagen or any other ingredient.

PRECAUTIONS: If sensitivity to the dressing develops, discontinue use. Dressings may be used with compression therapy under the supervision of a healthcare professional.

Features and Benefits

- Collagen and EDTA work together to target and deactivate excess MMPs, thus triggering the healing process in chronic wounds
- Sheet transforms into gel upon contact with exudates
- Cool, soothing effect of gelling action enhances patient comfort
- May be trimmed to exact wound size
- May be layered for the management of deep wounds
- Sterile, individually-wrapped sheets
- Not made with natural rubber latex



Satisfaction Guaranteed

For complete details, please visit mms.mckesson.com/mckesson-brands.

Never Compromise on Care™

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them—making it even easier for you to provide the best care for your patients.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.

DIRECTIONS FOR USE

1. Debride and irrigate the wound bed with an appropriate wound cleanser or distilled water before application.
2. Cut the dressing to fit the exact wound size. For heavily exudating wounds, apply the dressing directly to the wound bed. For dry wounds with very little exudate, moisten the wound bed with distilled water to begin the gelling process.
3. Cover the McKesson collagen dressing with an appropriate secondary dressing.
4. Change dressings daily or as good nursing practice dictates.

CARE AND REMOVAL:

1. Remove the secondary dressing with care. Gently remove the McKesson collagen dressing ensuring that the dressing removal does not damage any newly formed tissues.
2. Use an appropriate wound cleanser or distilled water to cleanse the wound site prior to applying a new McKesson collagen dressing as per application instructions.

NOTE: As with any medical device, always refer to the full instructions for use prior to application.



ORDERING INFORMATION

IMAGE	E1 #	MFR #	DESCRIPTION	DIMENSIONS	STERILITY	COLOR	QTY
	1171974	16-1850	Collagen Dressings	2 x 2 in	Sterile	White	1 ea, 10 ea/bx, 10 bx/cs
	1077005	16-1852	Collagen Dressings	4 x 4 in	Sterile	White	1 ea, 10 ea/bx, 10 bx/cs