

# DRUGS OF ABUSE DIP CARDS

The Consult drugs of abuse dip cards are rapid, all-in-one screening tests for the qualitative detection of multiple drugs and drug metabolites in urine at specific cutoff levels. This test provides results in only 5 minutes. It comes with an adulterant test strip that provides a semi-quantitative screen for oxidants, specific gravity, and pH in human urine which can help to assess the integrity of the urine specimen. This is a CLIA-waived test. You need a CLIA certificate of waiver to perform this test in waived settings.

## FEATURES AND BENEFITS

- CLIA-waived
- MFR #16-DTDC10P offers panel size up to 10 analytes
- MFR #16-DTDC12P offers panel size up to 12 analytes
- The test dip card provides screening for opiates at low-level cut offs (MOP300)
- Results in 5 minutes
- Minimum detection time after suspected drug use is 2-7 hours
- Detects one or more of the following drugs: AMP, BAR, BZO, BZD, COC, mAMP/MET, MTD, OPI, PCP, TCA, THC
- Not made with natural rubber latex



MFR#	DESCRIPTION	QTY
16-DTDC10P	Drugs of Abuse Dip Cards, 10-Drug Panel with Adulterants	1 pouch, 25 pouches/bx, 20 bx/cs
16-DTDC12P	Drugs of Abuse Dip Cards, 12-Drug Panel with Adulterants	1 pouch, 25 pouches/bx, 20 bx/cs

### Satisfaction Guaranteed

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit.

### Never Compromise on Care<sup>SM</sup>

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them—making it even easier for you to provide the best care for your patients.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.

# DRUGS OF ABUSE DIP CARDS

## Urine Screening Device Cutoff Levels

PANEL	CUTOFF (ng/mL)	DETECTION WINDOW
Amphetamine AMP	1000 ng/ml	2 Hours to 4 Days
Barbiturates BAR	300 ng/ml	2 Hours to 3 Days
Buprenorphine BUP	10 ng/ml	2 Hours to 3 Days
Benzodiazepine BZO	300 ng/ml	2 Hours to 4 Days
Cocaine COC	300 ng/ml	1 Hours to 4 Days
Methamphetamine MET	1000 ng/ml	2 Hours to 4 Days
Methadone MTD	300 ng/ml	3 Hours to 3 Days
Ecstasy MDMA	500 ng/ml	2 Hours to 4 Days
Morphine MOP	300 ng/ml	2 Hours to 3 Days
Oxycodone OXY	100 ng/ml	1 Hours to 2 Days
Phencyclidine PCP	25 ng/ml	4 Hours to 14 Days
Propoxyphene PPX	300 ng/ml	1 Hours to 3 Days
Nortriptyline TCA	1000 ng/ml	8 Hours to 7 Days
Marijuana/ Cannabinoids THC	50 ng/ml	2 Hours to 40 Days