GUARDIAN[®]

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User Instructions

Safety Summary: The following recommendations are made for the safe assembly of the raised toilet seat.

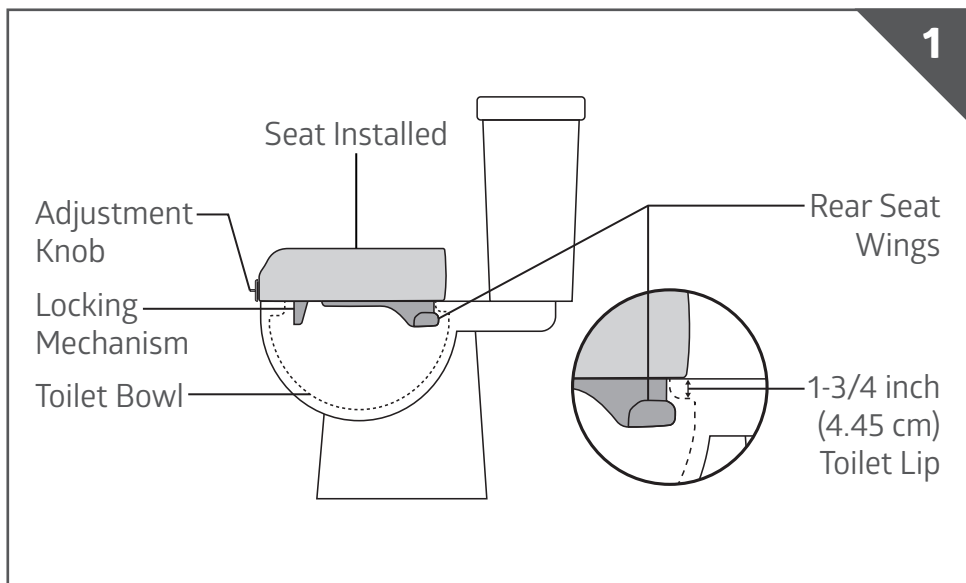
WARNINGS

- **DO NOT** assemble or use this raised toilet seat without first reading and understanding this instruction sheet. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this raised toilet seat. Otherwise, injury or damage may occur.
- Certain toilet fixtures will prevent the locking raised toilet seat from working properly and/or will damage the locking raised toilet seat. If the toilet bowl has any raised or protruding sections on its top or inner ring, the locking raised toilet seat should not be used.
- After assembly and **BEFORE** use, ensure that **ALL** attaching hardware is securely tightened.
- Always check to make sure that the raised toilet seat is properly and securely locked in place before using. Make certain that the knob that secures the raised toilet seat to the commode is tight at all times.
- Center your weight on the raised toilet seat. The seat may tip if you lean too far forward or to either side.
- Users with limited physical capabilities should be supervised or assisted when using the raised toilet seat.
- **DO NOT** exceed the maximum weight capacity of 400 lb (181 kg).
- Rotate the adjustment only by hand in order to prevent over torquing the assembly. Use of a wrench or other tool will break the screw assembly. **DO NOT FORCE** knob as this may damage the knob and/or the raised toilet seat.
- The Medline raised toilet seat is not designed for permanent installation. The unit should be loosened and retightened at least once a week to ensure a proper fit.
- If, for any reason, a proper fit in installation cannot be achieved, **DO NOT** use and contact retailer where purchased for return instructions.

Installation Instructions

The Medline raised toilet seat has been designed to accommodate a wide variety of toilet bowl types. However, certain bowl types will require the removal of the existing seat and/or lid to ensure proper fit when installed.

1. Lift the seat and lid of the toilet.
2. Rotate adjustment knob of the raised toilet seat counter clockwise until the paddle is far enough back to clear the inner rim of the bowl.
3. Place the raised toilet seat onto the bowl rim hooking the rear wings under edge of bowl. Check that the unit rests flat on the bowl rim. The knob must be at the front end of the toilet bowl.
4. Center the unit on the bowl rim and push the back as far as possible with your hands.
5. Rotate adjustment knob clockwise to tighten. Turn until resistance is felt, then turn an additional 1/8" until snug.

Note: Check all parts for shipping damage. In case of shipping damage, **DO NOT** use. Contact place of purchase for further instruction.

Figure 1

Cautions

- The adjustment bracket **MUST** be inside the toilet bowl and the raised toilet seat **MUST** be level and straight. The rear seat wings **MUST** be positioned under the rear lip of the toilet bowl. (See above figure)
- **DO NOT** use raised toilet seat if the toilet lip measurement exceeds 1-3/4 inches (4.45 cm) (See above figure), as the raised toilet seat will not hold and may cause injury.
- When the raised toilet seat is properly installed, the unit will sit flat on the top of the lip of the bowl and should fit securely against the inner rim of the bowl.

To Use Toilet with this Raised Toilet Seat

1. Carefully back up to the raised toilet seat until you feel the back of your knees touching it.
2. Slowly lower yourself onto the raised toilet seat while bending your elbows and knees.
3. Reverse steps (1) through (2) for getting up. Slowly raise yourself off of the toilet seat. Make sure you have your balance before releasing your grasp from the raised toilet seat.

To Remove the Raised Toilet Seat from the Toilet Bowl

1. Rotate the knob counterclockwise until the seat is loose.
2. Lift up the front of the raised toilet seat until the adjustment bracket clears the lip of the toilet bowl.
3. To completely remove the seat from the toilet bowl, pull the seat towards you at an angle.

To Clean the Raised Toilet Seat

This raised toilet seat can be easily cleaned with regular household cleaning products. **DO NOT** use abrasive powders or scouring pads.

1. Clean the seat at least once a week, using a non-abrasive detergent or non-abrasive cleaner with warm water.
2. Rinse with warm water and wipe with a non-abrasive cloth.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect not covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, which are subject to normal wear and replacement.

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©2023 Medline Industries, LP. All rights reserved. Medline is a trademark of Medline Industries, LP. /© Tous droits réservés, Medline Industries, LP, 2023. Medline est une marque de commerce de Medline Industries LP. /©2023 Medline Industries, LP. Todos los derechos reservados. Medline es una marca comercial de Medline Industries LP.