

DRIVE - MEDICAL SCALE CALIBRATION GUIDE

Step No.	Mode of press buttons	Pictures
1	If your calibration with LB weights, please Switch UINT to "LB"; If your calibration with KG weights, please Switch UINT to "KG"; In this guide, we go with LB weights for calibration.	
2	Press [HOLD] and [TARE] simultaneously, and keep pressing for 1 second. The display will be show "SETUP"	
3	Press [ON/OFF], [HOLD] and [TARE] simultaneously, and keep pressing for 1 second. The display will be show "LOAD0"	

4	Hanging the scale, make sure the scale stable and no shake, because need to guarantee to fix the accuracy ZERO Point; Press [TARE] to enter calibration, at the same time, scale be fixed with 0 point in loadcell. <i>Please notice, in the right picture, when we fix 0 point, the 0 point included with the down hooks.</i>	
5	Press [HOLD] to choose 200LB or 400LB weights to calibrate, if you choose 200LB, you need to calibrate with 200LB weights.	
6	Hanging 200LB weights until scale stable and no shake.	

7	Press [TARE] to finish the calibration. The display will be show "END"	
8	Press [TARE] again to exit the calibration. The scales show 200.0 LBS	



**Our Calibration instrument is
electric chain block**