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Safety Instructions

To ensure your safety in using the Medline rollator, the following safety information and all instructions must be followed.

GENERAL WARNINGS AND CAUTIONS:

- ⚠ **DO NOT use this product without first reading and understanding the instructions contained in this manual.** If you are unable to understand the warnings, cautions, or instructions, contact a healthcare professional, dealer, or technician before use. Failure to do this may result in bodily injury or damage to the device.
- ⚠ **DO NOT** self-propel the rollator while seated.
- ⚠ Rollators are intended for individual use only and are **NOT TO BE USED AS A WHEELCHAIR**. Serious injury to the user and/or damage to the rollator's frame or wheels may result from improper use.
- ⚠ **DO NOT** use the rollator to walk backwards, down gradients or to climb stairs, curbs, or to go over obstacles. Serious risk of fall or injury may occur.
- ✓ **CAUTION:** The backrest is not intended to support full body weight. Avoid leaning while sitting as this may result in a fall. Use **EXTREME** caution when shifting weight while seated against the backrest. This may cause the rollator to tip.
- ✓ All wheels must be in contact with the floor and/or ground at **ALL TIMES**. This will ensure the rollator is properly balanced.
 - DO NOT** exceed the maximum weight capacity (including bag contents) or serious injury could result: **Maximum weight capacity: 300 lb (136 kg)**
 - Items should be placed completely in the tote bag and should not protrude. Excess weight may cause the rollator to tip, resulting in bodily injury or rollator damage.
 - The tote bag located under the seat has a weight limitation of 5 lb (2.26 kg).
- ✓ Before each use make sure the rollator frame is fully and securely opened.
- ✓ Periodically wipe down the rollator with a soft cloth and non abrasive cleaner.

SEAT WARNINGS:

- ⚠ The brakes **MUST** be in the locked position **BEFORE** sitting on the seat.
- ⚠ **DO NOT** sit on the rollator while it is being loaded into a vehicle or while it is being transported in a vehicle.
- ⚠ **DO NOT** use the seat to transport people or objects.
- ⚠ **DO NOT** sit on the seat when the rollator is on an incline.
- ⚠ The rollator was designed to be used as a walking aid. **DO NOT** use the rollator in a manner that could cause personal injury; such as a scooter, a knee walker, or a jumping/hopping aid.
- ⚠ **DO NOT** stick fingers or hands near the seat slide. Be careful when folding

up and unfolding the rollator. Make sure that clothing and body parts are not pinched between the folding parts of the rollator.

- ✓ Observe the minimum insertion depth of the handholds.
- ✓ Each brake affects only one wheel.
- ✓ Do not use aggressive or caustic detergents to clean the rollator.
- ✓ Check that the rollator is fully intact each time before using it. Check that the brake is working.
- ✓ The surfaces of the rollator can heat up when exposed to direct sunlight. There is a risk of injury.

BRAKE WARNINGS:

- ⚠ Check the brakes for proper operation. Make sure that all parts are secure and that the moving parts are in good working order.
- ⚠ Wheels should roll easily and not wobble.
- ⚠ When rising from a lower position, **DO NOT** use the rollator to provide assistance to the user. If the rollator is used to pull the user up to a standing position, the rollator may move even if the brakes are in the locked position and this could result in a fall.
- ⚠ **DO NOT** push the rollator while brakes are enabled. Brakes are to be used to keep the wheels from rolling. However, it may not prevent the rollator from moving if it is pushed.

What's Included:

- A** 2 Five Point Knobs
- B** 4 Five Point Knob with Screw
- C** 2 Screws & Curved Washers

- D** 4 C clips
- E** Bag
- F** Rollator Frame
- G** 2 Handle Bars with Brake Assembly

- H** Backrest
- I** 2 Front Wheels
- J** User Instructions and Warranty Guide

This guide applies to the following rollator:

Item Number	Description	Frame Color	Weight Capacity
MDS868SKNM	Rollator, Basic Steel, Microban®	Navy	300 lb (136 kg)
MDS868SKWM	Rollator, Basic Steel, Microban®	White	300 lb (136 kg)

First: Open the Rollator

1

Remove the rollator components from shipping carton. Then, in an upright position, unfold the rollator by pushing down on the folding bracket until the frame is in full extension.

2

Place the rollator on its side.

IMPORTANT NOTE:

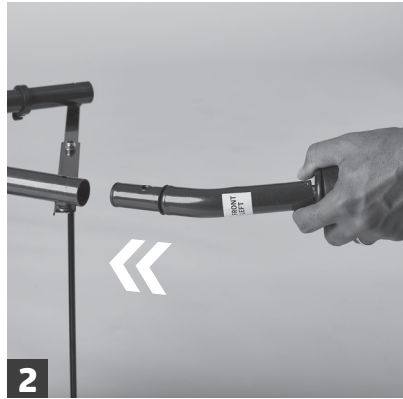
Proceed with caution when opening the rollator as there are areas around the moving parts that could potentially pinch fingers.

Now: Install the Wheels

Front Wheels

Remove the four black plastic plugs from the outer tubes and discard.

Attach front legs without brakes to the front of the rollator. The legs are left and right specific, make sure the threaded insert is facing outwards.

Insert the 5 point knob with screw and tighten. (Hand Tighten only.) Insert the C-clip just above the 5 point knob. Repeat steps 1-3 for the other front leg extension without brakes.

Rear Wheels & Brakes

With the rollator still on its side, attach rear wheels with brakes. Make sure that the rear wheels and brake shoe are on the outside of the rollator and brake handle (See above). Use the 5 point screw to thread the rear legs in place.

Insert the C-Clip just above the 5 point knob. Repeat steps for other rear leg. Ensure all C-clips are secure and 5 Point knobs are fully tighten prior to each use.

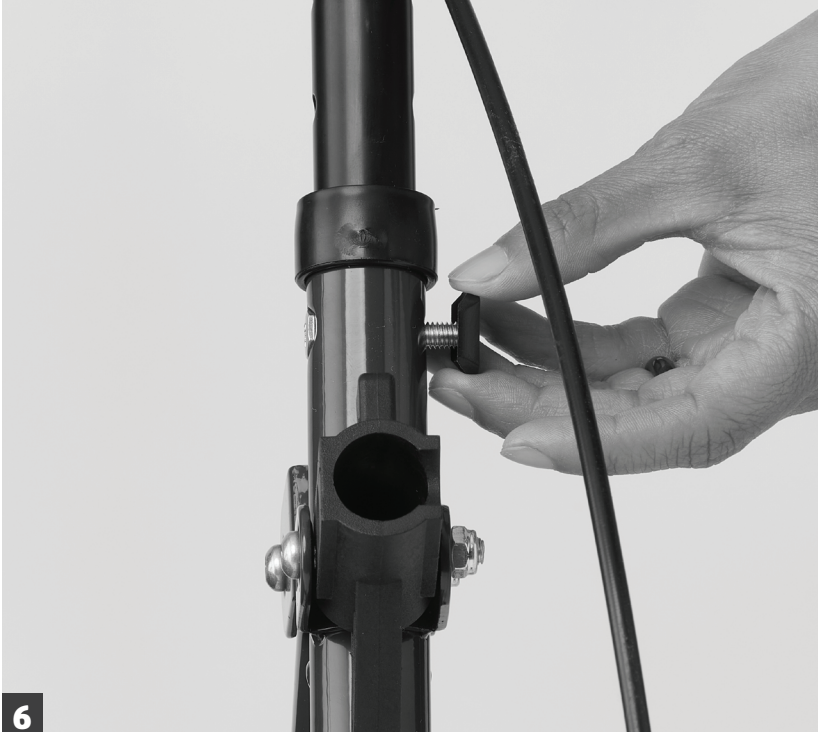
4

Stand the rollator upright and insert the brake handle assembly. Brake cables must be on the outside of the handle bars and frame.

5

Insert the screw into the 6 sided hole in the frame. Make sure the screw head recesses into the frame.

6

Slide the curved washer onto the other end of the screw with the curved side facing the frame.

7

Screw on the 5 point knob.

8

Repeat steps 4–7 on the other side. Make sure the push handles are set at the same height and are fully tightened before use. (Hand tighten only)

Finally Install Backrest

Insert the backrest so that the curved portion faces the front of the unit. Make sure that the button snaps firmly into the hole so the back is secure.

Press excess brake cable into the cable clips. Attach the bag by looping the hook and loop straps around the frame and pressing firmly on the bag.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

Limited lifetime warranty on frame; 6 months on parts:

- Hand brakes
- Brake cables
- Push-buttons
- Backrest
- Brakes
- Seat
- Wheels (front and rear)
- Bag

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Microban® antimicrobial product protection is limited to the product itself and is not designed to protect users from disease causing microorganisms or as a substitute for normal cleaning and hygiene practices.

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Medline México
01-800-831-0898 | medlinemexico.com | mexico@medline.com

Follow us
Suivez-nous
Síguenos

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2022 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP. Microban® is a registered trademark of Microban Products Company.

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