Manual Seated Scooter, Black
Trottinette manuelle avec siège, Noir
Escúter manual para uso sentado, Negro

**Safety
Guidelines**
**Consignes
de sécurité**
**Pautas de
seguridad**

Weight Capacity
Capacité de poids de
Capacidad de peso de
300 lbs (136 kg)

REF MDS86000SS

- » Before using the MDS86000SS, you should be trained by a healthcare professional or dealer.
- » **DO NOT** use the MDS86000SS by yourself until you have been properly trained, you have practiced with someone and you feel confident you can maneuver by yourself.
- » Engage the lock and practice bending, reaching and transfers on and off the MDS86000SS.
- » **NEVER** attempt a maneuver that has not been practiced.
- » Be aware of your surroundings when operating the MDS86000SS. Look for hazards and avoid them.
- » **BEFORE** each use, make sure the seat pad is in the locked position. Check the brakes for proper operation.
- » **DO NOT** exceed the maximum weight capacity listed on the device and in the assembly guide or serious injury could result.
- » **DO NOT** perform any adjustments while the MDS86000SS is in use.
- » **ALL WHEELS** must be in contact with the floor at **ALL TIMES**.
- » When turning, use **EXTREME CAUTION** and only turn when one foot is on the ground.
- » **ALWAYS** keep both hands on the handle grips when in use.

Cautions

- » **DO NOT** lean forward, backward or to either side while using.
- » **DO NOT** lean forward while applying the brakes.
- » **DO NOT** operate on loose gravel or uneven surfaces.
- » **DO NOT** travel over large cracks or fissures.
- » Stop and move forward slowly when encountering cracks or separations in walkway surfaces.
- » **DO NOT** operate in excess of 3 miles per hour (slow-to-moderate walking speed).
- » **DO NOT** turn while coasting.
- » **DO NOT** turn unless one foot is on the ground.
- » **DO NOT** turn at speeds over 1 mph (very slow walking speed).
- » **DO NOT** operate on inclines above 15 degrees (slight incline).
- » **DO NOT** travel over drops or barriers such as curbs, steps or parking barriers.
- » Always check the fasteners on the tiller, wheels, brakes, and pad

for tightness before use.

- » **DO NOT** use the MDS86000SS for anything other than its intended use. This is a walking aid **ONLY**.
- » **DO NOT** use the seat to transport objects.
- » **DO NOT** sit on the MDS86000SS while it is being loaded into a vehicle or while it is being transported in a vehicle.
- » **DO NOT** sit, stand or lean on the steering mechanism. The steering mechanism is not weight bearing.
- » **DO NOT** sit on the MDS86000SS unless the seat pad is in a locked position.
- » **DO NOT** hang anything from the MDS86000SS handles or frame at any time other than a Medline provided basket or bag. This may cause the walker to tip.
- » **DO NOT** put more than 5 lbs in the bag or basket.
- » **DO NOT** use the MDS86000SS if the brake system is not working properly.

Warranty

Your Medline Product is warranted to be free of defects in materials and workmanship for the lifetime of the consumer. This device was built to precise standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the material and workmanship of our products and assures years of convenient, dependable service. This warranty does not extend to non-durable components such as handgrips, pushbuttons, leg riggings and any other wearable part which carry a 6 months warranty. In the event of a defect or failure covered by this warranty, contact us at **1-800-MEDLINE**. Medline will replace your Medline product.

This warranty does not extend to damage or failure occurring as a result of owner misuse or negligence, normal wear and tear, or other causes unrelated to the manufacture and design of the product. This express warranty is Medline's sole and exclusive warranty and your sole remedy in the event of default or failure. Medline is not responsible for any consequential or incidental damages or costs arising from use of the warranted products, or any defect therein or failure thereof. Medline disclaims all other warranties including but not limited to the implied warranties of merchantability and fitness for a particular purpose.

Contact us at **1-800-MEDLINE**, Monday – Friday 8:00 am to 5:00 pm CST if you encounter any problems or for information regarding repairs or maintenance.

Maintenance

1. Clean the frame/hand grip and seat pad with a mild soap, detergent or household cleaner. Do not use an organic solvent (Such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause damage to the product. Thoroughly dry the product **IMMEDIATELY** with a towel.
2. Check the hardware prior to each use to ensure that all components are properly tightened and/or fastened.
3. Check to see if tires need inflating prior to use. Tire pump not included.
4. On a monthly basis or as necessary; check the condition of the frame and ensure that it is free of defects.

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Some products may not be available for sale in Mexico or Canada. We reserve the right to correct any errors that may occur within this brochure. © 2022 Medline Industries, LP. All rights reserved. Medline is a registered trademark of Medline Industries, LP.