**Economy Knee Walker, Junior, Red**  
**Déambulateur économique avec**  
**repose-genou, junior, rouge**  
**Andador de rodilla económico,**  
**pequeño, rojo**

**Safety**  
**Guidelines**  
**Consignes de**  
**sécurité**  
**Pautas de seguridad**

**REF** MDS86000EJR

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- » Before using the Knee walker, you should be trained by a healthcare professional or dealer.
- » **DO NOT** use the Knee walker by yourself until you have been properly trained, you have practiced with someone and you feel confident you can maneuver by yourself.
- » Engage the lock and practice bending, reaching and transfers on and off the Knee walker.
- » **NEVER** attempt a maneuver that has not been practiced.
- » Be aware of your surroundings when operating the Knee walker. Look for hazards and avoid them.
- » **BEFORE** each use, make sure the kneepad is in the locked position. Check the brakes for proper operation.
- » **DO NOT** exceed the maximum weight capacity listed on the device and in the assembly guide or serious injury could result.
- » **DO NOT** perform any adjustments while the economy knee walker is in use.
- » **ALL WHEELS** must be in contact with the floor at **ALL TIMES**.
- » When turning, use **EXTREME CAUTION** and only turn when one foot is on the ground.
- » **ALWAYS** keep both hands on the handle grips when in use.

## Cautions

- » **DO NOT** lean forward, backward or to either side while using.
- » **DO NOT** lean forward while applying the brakes.
- » **DO NOT** operate on loose gravel or uneven surfaces.
- » **DO NOT** travel over large cracks or fissures.
- » Stop and move forward slowly when encountering cracks or separations in walkway surfaces.
- » **DO NOT** operate in excess of 3 miles per hour (4.8 km/h) (slow-to-moderate walking speed).
- » **DO NOT** turn while coasting.
- » **DO NOT** turn unless one foot is on the ground.
- » **DO NOT** turn at speeds over 1 mph (1.6 km/h) (very slow walking speed).
- » **DO NOT** operate on inclines above 15 degrees (slight incline).
- » **DO NOT** travel over drops or barriers such as curbs, steps or parking barriers.
- » Always check the fasteners on the tiller, wheels, brakes, and pad for tightness before use.

- » **DO NOT** use the Knee walker for anything other than its intended use. This is a walking aid **ONLY**.
- » **DO NOT** use the seat to transport people or objects.
- » **DO NOT** sit on the Knee walker while it is being loaded into a vehicle or while it is being transported in a vehicle.
- » **DO NOT** sit, stand or lean on the steering mechanism. The steering mechanism is not weight bearing.
- » **DO NOT** sit on the Knee walker unless the brakes and padded knee rest are in a locked position.
- » **DO NOT** self propel the Knee walker while seated.
- » **DO NOT** hang anything from the Knee walker handles or frame at any time other than a Medline provided basket or bag. This may cause the walker to tip.
- » **DO NOT** use the Knee walker if the brake system is not working properly.

## Warranty

Your Medline Product is warranted to be free of defects in materials and workmanship for the lifetime of the consumer. This device was built to precise standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the material and workmanship of our products and assures years of convenient, dependable service. This warranty does not extend to non-durable components such as handgrips, pushbuttons, leg riggings and any other wearable part which carry a one (1) year warranty. In the event of a defect or failure covered by this warranty, contact us at **1-800-MEDLINE | Medline Canada 1-800-268-2848**. Medline will, at its option, repair or replace your Medline product.

This warranty does not extend to damage or failure occurring as a result of owner misuse or negligence, normal wear and tear, or other causes unrelated to the manufacture and design of the product. This express warranty is Medline's sole and exclusive warranty and your sole remedy in the event of default or failure. Medline is not responsible for any consequential or incidental damages or costs arising from use of the warranted products, or any defect therein or failure thereof. Medline disclaims all other warranties including but not limited to the implied warranties of merchantability and fitness for a particular purpose.

Contact us at **1-800-MEDLINE | Medline Canada 1-800-268-2848**, Monday – Friday 8:00 am to 5:00 pm CST if you encounter any problems or for information regarding repairs or maintenance.

# Maintenance

1. Clean the frame/hand grip and knee pad with a mild soap, detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause damage to the product. Thoroughly dry the product **IMMEDIATELY** with a towel.
2. Check the hardware prior to each use to ensure that all components are properly tightened and/or fastened.
3. On a monthly basis or as necessary; check the condition of the frame and ensure that it is free of defects.

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Some products may not be available for sale in Mexico or Canada. We reserve the right to correct any errors that may occur within this brochure. © 2022 Medline Industries, LP. All rights reserved. Medline is a registered trademark of Medline Industries, LP.